



Instructions for Use: Post Procedure Pre-Treatment Kit

Brand Name of Product	Post Procedure Pre-Treatment Cleaning Kit
Generic Name of Product	Cleaning Kit
Product Code Number(s)	PPPTK-001
Intended Use	For rinsing and wiping of flexible endoscopes at the point of use/bedside.
Range of Applications for Product	Pre-cleaning flexible endoscopes after procedures before transporting to the decontamination area.
Key Specifications of Product	<ol style="list-style-type: none"> 1. 800 mL of micron filtration water. 2. Flat sponge 5”L x 4”W x 0.5”H (127mm x 102mm x 12.5mm), high density, 100% medical grade, biodegradable foam. 3. Bag is notched, easy tear open packaging. 4. Stable, wide, clear pouch for easy fluid transfer and direct view of the solution.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Flushing and wiping down a flexible endoscope after a procedure following the endoscope manufacturer’s IFU.
Preparation for Use	Open package using the tear notches that are located on both sides of the pouch. Grab the top part of the bag at the notch and tear across to open the bag.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Wearing appropriate PPE, place a gloved hand inside the bag and squeeze excess fluid out of the flat sponge. Remove the sponge from the pouch. 2. Starting at the control section of the scope, place the flat sponge around it and wipe down the length of the insertion tube from the control section to the distal tip to remove any excess bioburden of the surface. 3. Take the distal end of the insertion tube, insert it into the pouch and submerge into the solution. 4. Suction fluid and air through the endoscope instrument suction channel as instructed by the endoscope manufacturer. 5. After several cycles, remove the insertion tube from the solution allowing the solution to drain from the channel by lifting the tube out of the solution so it drains out. 6. Follow the endoscope manufacturer’s IFU for any required additional steps for pre-cleaning. 7. Place the endoscope into a transport container and transport to the decontamination area.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	Store away from direct light. With extended light exposure during storage, the foam sponge may change color.
Disposal	Discard the sponge, bag and fluid using approved methods of disposal.

Reprocessing Instructions

Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	ProFormance™ Brochure, ProFormance™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-05-18 Suzanne Latta