



Instructions for Use: Cleanroom Sterile Wipes

Brand Name of Product	Sterile Wipes
Generic Name of Product	Prepackaged Sterile Wipes
Product Code Number(s)	SSW-001, XX6700
Intended Use	For use in the reprocessing of flexible endoscopes in compliance with the endoscope manufacturer's instructions for use.
Range of Applications for Product	Sterile non-linting wipes used in compliance with flexible endoscope reprocessing IFUs.
Key Specifications of Product	<ul style="list-style-type: none"> • Sterile • Absorbent • Non-linting • Non-woven polyester cellulose blend

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	For use in the reprocessing of flexible endoscopes in compliance with the endoscope manufacturer's instructions for use.
Preparation for Use	Follow the endoscope manufacturer's IFU for the use of sterile wipes during reprocessing procedures. A partial list of those procedures is listed below.
Diagrams (drawings, pictures)	
Steps for Use of Product	<p>Possible uses include (consult the endoscope manufacturer's IFU for detailed instructions):</p> <ul style="list-style-type: none"> • Drying the external surfaces after high level disinfection • Covering the distal end and the control section of the endoscope • Prior to sterilization, drying external surfaces of the endoscope and accessories • Wiping the basin and connectors of an AER to dry • Do not wipe in circular motions. For efficient wiping, wipe in one direction using parallel, overlapping strokes • Dispose of wipes after each individual use (single use)
	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	Single use only.
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A

Sterilization	N/A
Storage	N/A
Related Healthmark Products	Non-linting Wipes
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	ANSI/AAMI/ISO 11137:2013
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-07-26 Suzanne Latta