



Instructions for Use: Pre-Rinse Shower Unit

Brand Name of Product	Pre-Rinse Shower Unit
Generic Name of Product	Shower Unit
Product Code Number(s)	300718, 300719, 301189, 301190, 313116, 313323
Purpose of Product	For safe and effective rinsing of gross contaminants from surgical instruments.
Range of Applications for Product	Surgical instruments and other medical devices
Key specifications of product	<ul style="list-style-type: none"> • Polished Chrome Plated Brass • Single Hole Deck Mount Pantry Faucet • 1/4 Turn Compression Valves w/ Check Valves • Lever Handles w/ SANIGUARD® Treatment • S/S Flex Hose w/ Santoprene Reinforced Hose • Stainless Steel Strain Relief – Patented • O-Ring Seals - Patented • Standard Full Spray Pattern • 1.2 GPM Spray Face Included • Hold open ring and hose hook included • Commercial Specification Grade • Gaskets and Seals Rated Commercial Hot 180°F • 1/2" NPS Female Inlets • Operating Temperature: 40 - 180°F • Operating Pressure: 15 - 125psi

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Rinsing of gross contaminates from surgical instruments and equipment.
Preparation for Use	Be sure to consult the IFU of the medical device manufacturer to insure that the device should be exposed to spray washing.
Diagrams (drawings, pictures):	
Steps for Use of Product	<ul style="list-style-type: none"> • Place instruments soiled and with gross debris under spray unit. • Rinse thoroughly with spray to remove gross debris and soil before further reprocessing.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	<ul style="list-style-type: none"> • It is usually recommended to use water from the facility's cold water source for the initial rinse in order to avoid denaturing proteins and other organics. Consult the device manufacturer's IFU.

	<ul style="list-style-type: none"> • The Shower Unit is not a medical product and is therefore not to be used as part of medical treatments. • When in operation, the Shower Unit must not be pointed at parts or orifices of the body because the compressed water or air jet may be harmful. • Be sure to wear proper PPE according to facility and industry guidelines.
Disposal	

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	
Storage:	
Additional Information:	Options include deck mount, splash mount, faucet add-on, wall bracket and an optional 36” long hose.

Related Healthmark Products	
Other Product Support Documents	Prosys Brochure, Prosys Price List
Reference Documents	Complies with: <ul style="list-style-type: none"> • ANSI A112.18.1M • CSA B125 • NSF 61 / 9 • ASTM F2324
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2012-08-13 msmith