
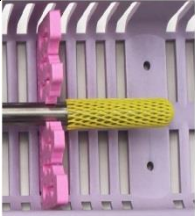
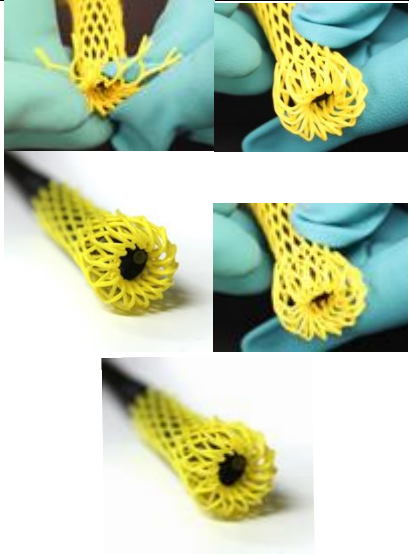
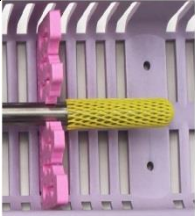
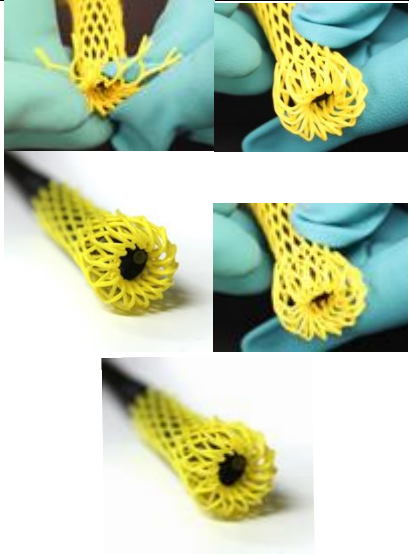
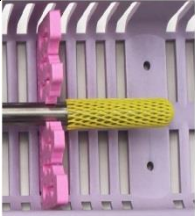


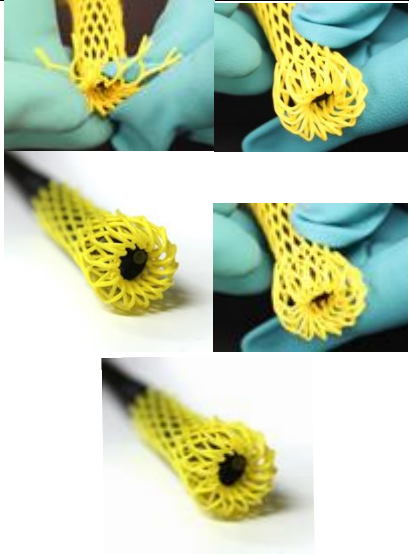

Instructions for Use: Mesh Tip Protectors

Brand Name of Product	Mesh Tip Protectors
Generic Name of Product	Tip Protectors
Product Code Number(s)	CSW-03-2.0, CSW-04-4.0
Intended Use	To protect delicate items during Vaporized Hydrogen Peroxide Sterilization and/or storage.
Range of Applications for Product	Small to large tipped items.
Key Specifications of Product	<ul style="list-style-type: none"> • Compatible with VH202 Sterilization including STERRAD® and V-Pro® systems. • Mesh polyethylene material to promote sterilant circulation. • Available in closed or open-ended styles. • Single-use

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product					
Description of Use(s)	To protect delicate instruments during VH2O2 sterilization and storage.				
Preparation for Use					
Diagrams (drawings, pictures)	<table border="1" style="width: 100%;"> <tr> <td style="text-align: center;">  </td> <td style="text-align: center;">  </td> </tr> <tr> <td style="text-align: center;">Closed Style Tip</td> <td style="text-align: center;">Open Style Tip</td> </tr> </table>			Closed Style Tip	Open Style Tip
					
Closed Style Tip	Open Style Tip				
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place protector tip on the distal tip. The mesh style tip protector will provide some flexibility to expand to the instrument size. 2. If open style, optionally roll mesh tip ends of the protector to provide cushion support for the exposed tip (see open style tip pictured above). 				
Interpretation of Results					
Contraindications of Test Results					
Documentation					
Special Warnings and Cautions	For items undergoing HLD, apply the tip protector after all processing procedures.				
Disposal	Dispose of in compliance with facility guidelines.				

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	

Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	Compatible with VH2O2 systems, including STERRAD® and V-Pro®.
Storage	
Additional Information	

Related Healthmark Products	Tip Protectors, Vented Instrument Guards, Duo-Vented Instrument Guards, Cylindrical Style, Instrument Guards.
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price
Reference Documents	List Mesh Style Tip Protectors-Validation for Compatibility with Hydrogen Peroxide Sterilization: http://www.healthmark.info/InstrumentCare/TipProtectors/Mesh_style_tip_protectors_white_paper_Jan_2016_KK.pdf
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2020-08-05msmith