



Instructions for Use: Vinyl Instrument Guards

Brand Name of Product	Vinyl Instrument Guards
Generic Name of Product	Tip Protectors
Product Code Number(s)	09-1-011, 09-1-012, 09-1-013, 09-1-014, 09-1-015, 09-1-016, 09-1-017, 09-1-018, 09-1-019, 09-1-020, 09-3-031, 09-3-032, 09-2-033, 09-3-034, 09-3-035, 09-03-037, 09-03-040, 09-07-011, 09-7-012, 09-7-013, 09-7-014, 09-07-15
Purpose of Product	Vinyl instrument guards are designed to enhance sterilization and protection for instruments during the sterilization procedure. The vents allow the flow of full-impact sterilization while providing protection to the instruments tips and edges from breaking and/or chipping.
Range of Applications for Product	For trocars, probes and other small tipped instruments.
Key specifications of product	<ul style="list-style-type: none"> • Compatible with steam sterilization. • Variety of colors (For Vented and Duo-Vented) • Vented, Duo-Vented and Cylindrical styles

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	To be placed on small tip surgical instruments
Preparation for Use	Fit each instrument with the appropriate size tip.
Diagrams (drawings, pictures):	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place protector tip on appropriate sized instrument prior to sterilization. 2. Package instruments in sterile packaging device (e.g., pouch, wrapped tray, closed container). 3. Place package in sterilizer (steam).
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions

Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	Compatible with steam sterilization. Recommended cycle is 4 minutes at 270°F or 3 minutes at 275°F in a pre-vac sterilizer.
Storage:	
Additional Information:	

Related Healthmark Products	
Other Product Support Documents	Sterilization Methods For Color Tip Protectors
Reference Documents	Prosys Price List, Prosys Brochure
Customer Service contact:	Healthmark Industries Company, Inc

	33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com
--	---

2015-01-21 msmith