






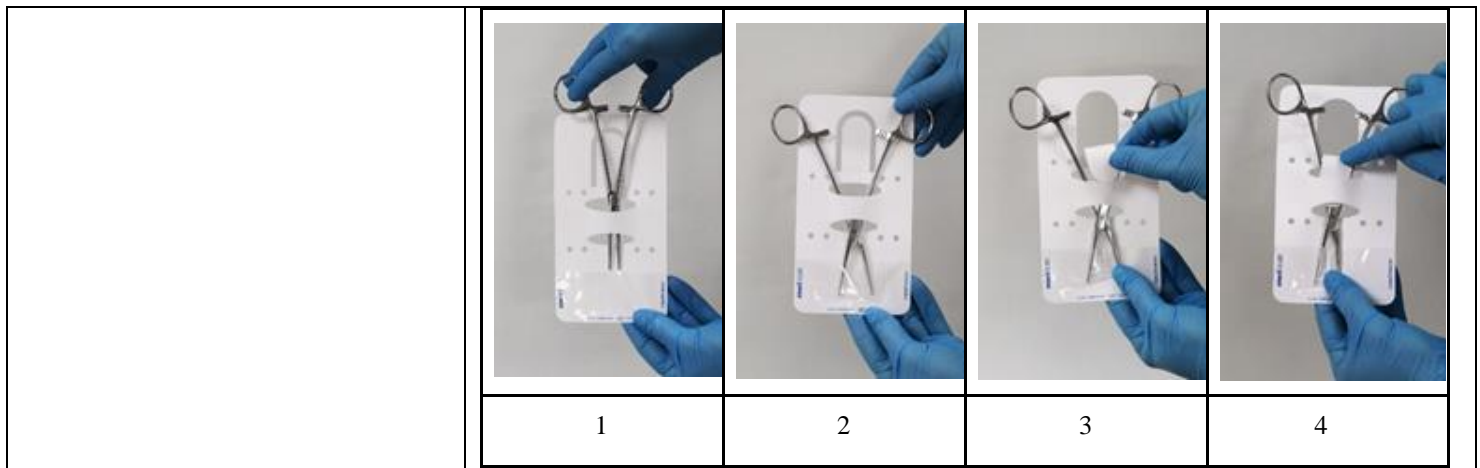


Instructions for Use: Window Style Tip Protectors

Brand Name of Product	Window Style Tip Protectors
Generic Name of Product	Tip Protectors
Product Code Number(s)	15.100.055, WSTP-001
Purpose of Product	Designed to protect tips during sterilization, transport and storage.
Range of Applications for Product	For variety of small sizes.
Key Specifications of Product	<ul style="list-style-type: none"> • Single-use • Paper: 250-gram wood free white blanc, allowing excellent performance • Pouch: Transparent pouch made of polyethylene-polypropylene laminate • Ink: Non-toxic water-solvent based ink (pH in between 9.0 and 9.5 at 25°C)

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	Temperature: 10°C - 40°C Relative humidity: 20% < RH < 80% Keep away from direct sunlight / keep dry
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use(s)	To protect tips during sterilization, transport and storage.
Preparation for Use	
Diagrams (drawings, pictures)	<p>Item Code- 15.100.055</p> <div style="display: flex; flex-wrap: wrap;"> <div style="width: 33%; text-align: center;">  <p>1</p> </div> <div style="width: 33%; text-align: center;">  <p>2</p> </div> <div style="width: 33%; text-align: center;">  <p>3</p> </div> <div style="width: 33%; text-align: center;">  <p>4</p> </div> <div style="width: 33%; text-align: center;">  <p>5</p> </div> </div> <p>Item Code- WSTP-001</p>



Steps for Use of Product	<p>Item Code- 15.100.055</p> <ol style="list-style-type: none"> Place the tip through top slotted insert. Fig. 1 & Fig. 2 Continue sliding the item down the backside of the protector. Tip should emerge through the second slotted insert in front of the protector. Fig. 3 Make sure tip slides inside of transparent window pouch. Fig. 4 Keep sliding item down until it is secure. Fig. 5 <p>Item Code- WSTP-001</p> <ol style="list-style-type: none"> Place the tip through both slots. Fig. 1 Continue sliding the item down the backside of the protector. Make sure tip slides inside of transparent window pouch. Fig. 2 Keep sliding item down until it is secure. Fold the flap over and secure underneath the top slot to ensure the item is kept open. Fig. 3. Fig. 4
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	Single-use, dispose after use in accordance with facility policy.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	<ul style="list-style-type: none"> ● Compatible with steam sterilization. Recommended cycle is 4 minutes at 132°C or 3 minutes at 135 °C in a pre-vacuum sterilizer. ● Material Compatibility <ul style="list-style-type: none"> ○ Steam up to 145°C ○ EO
Storage	
Additional Information	

Related Healthmark Products	Transparent Tip Protectors, Vented Instrument Guards, Duo-Vented Guards, Cylindrical Style Instrument Guards
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026

	1-586-774-7600 healthmark@hmark.com hmark.com
--	---

2020-08-06 msmith