



Instructions for Use: Nic-Nac Washer Baskets & Balls

Brand Name of Product	Nic-Nac Washer Baskets & Balls
Generic Name of Product	Washer baskets for small items
Product Code Number(s)	NIC000, NIC001, 2100-H, NIC002, NIC003, NIC MICRO 1, NIC MICRO 2A, NIC MICRO 2, NIC MICRO 3, CIL-004.
Purpose of Product	To hold small parts and instruments when being washed inside of an automated instrument washer, preventing damage and loss.
Range of Applications for Product	Small parts and instruments that could be lost blown around, damaged or lost in an automated instrument washer.
Key Specifications of Product	<ol style="list-style-type: none"> 1. Manufactured from stainless steel. 2. Hinged lid for safe containment of small parts. 3. Mesh design to allow thorough exposure to cleaning and rinsing solution within an automated instrument washer.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Place in the washer basket small surgical instruments and parts that often get lost or damaged inside of an automated instrument washer.
Preparation for Use	
Diagrams (drawings, pictures)	
Steps for Use of Product	<ul style="list-style-type: none"> ● Place small instruments in basket ● Close lid of basket ● Place into instrument washer for cleaning
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	Compatible with steam sterilization. Recommended cycle is 4 minutes at 270°F (132°C) or 3 minutes at 275°F (135°C) in a pre-vac sterilizer. Maximum temperature is 275°F (135°C) for 30 minutes.
Storage	
Additional Information	

Related Healthmark Products	Washer Baskets
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List

Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-08-31 msmith