



<b>Brand Name of Product</b>	Humipak®
<b>Generic Name of Product</b>	Self-seal pouch with absorbent layer for transporting and storing instruments in humid environment.
<b>Product Code Number(s)</b>	HPSS2035, HPSS4243, HPSS4267, HPSS6577
<b>Purpose of Product</b>	To provide a humid and moist environment delaying the drying of bio-contaminants' on medical items such as surgical instruments.
<b>Range of Applications for Product</b>	Surgical trays, flexible endoscopes and other contaminated medical devices.
<b>Key specifications of product</b>	<ul style="list-style-type: none"> <li>• Self-seal strip.</li> <li>• A special absorbent layer which when wetted creates a humid atmosphere within the pouch.</li> <li>• Two layers of transparent film (containing the special absorbent layers) form the pouch.</li> <li>• The transparent film, in addition to maintaining the moist atmosphere, allows the contents to be clearly observed.</li> <li>• 4 sizes: <ul style="list-style-type: none"> <li>○ Small, Half Din, Full Din, Double Din</li> </ul> </li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	Store in a clean, dry place at room temperature
<b>Packaging Conditions</b>	
<b>Shelf Life</b>	5 year shelf life from the date of manufacture

<b>Instructions for Using Product</b>	
<b>Description of Use (s)</b>	Placing surgical instruments and other products into the Humipak® after use will keep items moist longer delaying the drying of bio-contaminants before decontamination commences.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures):</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Pick the correct size of Humipak® pouch for the items that are to be kept moist until reprocessing.</li> <li>2. Add the specified amount of water per the directions found on the Humipak® pouch.</li> <li>3. Water will wick and be absorbed throughout the Humipak® pouch because of the special absorbent layer between the transparent films.</li> <li>4. Place items in the pouch.</li> <li>5. Peel off protective strip from adhesive band.</li> <li>6. Fold over the adhesive strip and press down firmly from the center outwards to insure a watertight seal.</li> <li>7. Send to the appropriate area for reprocessing as soon as possible.</li> <li>8. The Humipak® is NOT a sharps container and is NOT puncture proof. Be sure to follow your facilities policy for handling contaminated reusable sharps in compliance with OSHA Guidelines.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• Items placed in the Humipak® should be cleaned as soon as possible.</li> <li>• Single-use only.</li> <li>• Do not reprocess or sterilize.</li> <li>• Use PPE when opening the Humipak®</li> <li>• The Humipak® is NOT a sharps container and is NOT puncture proof. Be sure to follow your facilities policy for handling contaminated reusable sharps in compliance with OSHA Guidelines.</li> </ul>

<b>Disposal</b>	Treat used Humipak® as a biohazard and dispose of in accordance with facility policy for contaminated items.
-----------------	--

<b>Reprocessing Instructions</b>	
<b>Point of use:</b>	
<b>Preparation for decontamination:</b>	
<b>Disassembly Instructions:</b>	
<b>Cleaning – Manual:</b>	
<b>Cleaning – Automated:</b>	
<b>Disinfection:</b>	
<b>Drying:</b>	
<b>Maintenance, inspection, and testing:</b>	
<b>Reassembly Instructions:</b>	
<b>Packaging:</b>	
<b>Sterilization:</b>	
<b>Storage:</b>	
<b>Additional Information:</b>	

<b>Related Healthmark Products</b>	Decontam Gloves, SST Pump (Item #IP5-4, 538DP-Applied, SST-Latch, Splash –Proof LTD, SST Tray Stands
<b>Other Product Support Documents</b>	SST Instrument Retrieval Brochure, SST Instrument Retrieval Price List, Humipak® Pocket Brochure.
<b>Reference Documents</b>	
<b>Customer Service contact:</b>	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com

2017-12-21 msmith