



Instructions for Use: Beard Net

Brand Name of Product	Beard Net
Generic Name of Product	Beard Net
Product Code Number(s)	BN300
Intended Use	To cover the beard to limit as a source of possible contamination in the prep and pack area and decontamination area.
Range of Applications for Product	N/A
Key Specifications of Product	<ul style="list-style-type: none"> ● One size fits most ● Super fine mesh nylon with small holes ● Single-use

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	100 in a box
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To cover the beard to limit as a source of possible contamination.
Preparation for Use	Take one beard net from the box.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place hands around the band of the beard net. 2. Place the beard net over the head and around facial hair. 3. The band is placed around the back of the head and above the ears. 4. Make sure all facial hair is completely covered by the beard net. 5. To remove, with both hands take the band and lift over the head and pull the beard net away from face.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> ● One person, one use ● Not intended for use in the procedure areas.
Disposal	Single-use only

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A

Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	PPE Brochure, PPE Price List

Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com