

Instructions for Use: Cool Aids Vest

Brand Name of Product	Cool Aids Vest
Generic Name of Product	Vest
Product Code Number(s)	VEST-S, VEST-M, VEST-L, VEST-XL, VEST-XXL, VEST-XXXL, VEST-XXXXL CPX-001
Intended Use	To provide cooling of the core body temperature.
Range of Applications for Product	
Key Specifications of Product	<ul style="list-style-type: none"> • V-neck w/zipper closure • Lightweight, easy to activate, and durable • One set of four cooling inserts is included with each vest and one cooler bag • Includes inside pockets for cooling inserts

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	To provide cooling of the core body temperature.
Preparation for Use	<ul style="list-style-type: none"> • Activate by placing phase change packs in a refrigerator or freezer to cool. • Simply place phase change packs in freezer for 20 minutes. • Or place in the refrigerator for 40 minutes. • Phase change packs freeze at 58°F. • It is recommended to wear a thin undergarment such as a cotton T-Shirt under the vest for comfort.
Diagrams (drawings, pictures)	
Steps for Use of Product	<ul style="list-style-type: none"> • Place each phase change packs (4 total) inside the mesh pockets inside the vest. • Wait 10 minutes before putting on the vest. • Place each arm into the vest so it's secure around each shoulder. • Zip the vest completely up to your upper chest.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions

Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	<p><u>Vest</u></p> <ul style="list-style-type: none"> • Remove the phase change packs from the vest prior to manual cleaning. • Hand wash in warm water using a mild soap. Rinse. Hang to dry. • NOTE: Cleaning/washing of these products should be done in a clean area (ie locker room). <p><u>Phase Change Packs</u></p> <ul style="list-style-type: none"> • Clean with light soap and water
Cleaning – Automated	<p><u>Vest</u></p> <ul style="list-style-type: none"> • When using the washing machine, remove phase change packs from vest prior to cleaning. • Use light soap and gentle cycle.

	<p><u>Phase Change Packs</u></p> <ul style="list-style-type: none"> Wash phase change packs manually with light soap and water.
Disinfection	
Drying	Item should then be hung up in a clean area to dry. Keep in a clean area between uses.
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	PPE Brochure, PPE Price List
Reference Documents	
Customer Service Contact	<p>Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com</p>

2018-09-07 Ralph J Basile