



Instructions for Use: Disposable Jumpsuit with Hood and Shoe Covers

Brand Name of Product	Disposable Jumpsuit
Generic Name of Product	Jumpsuit
Product Code Number(s)	30122-M, 3022-L, 30122-XL, 30122-XXL, 30122-XXXL
Intended Use	To cover up person's outer wear from head to toe from coming into contact with items in the decontam area.
Range of Applications for Product	To protect the individual's outer clothing including hair and shoes from contamination.
Key Specifications of Product	<ul style="list-style-type: none"> • White spun-bound material • Elastic cuff and wrist

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use (s)	To keep outer clothes, hair and shoes from coming into contact with contaminated items.
Preparation for Use	<p>Donning:</p> <ol style="list-style-type: none"> 1. Unfold jumpsuit. 2. Place one foot into the leg inserting the foot into the shoe cover, then place other foot into the second leg of the shoe cover of the suit. 3. Pull jump-suit up and put each arm thru the sleeve of the suit until your hand comes out past the cuff of the sleeve. 4. Pull suit up and around the shoulders to cover entire body. 5. Place the hood on head to cover the hair entirely. 6. Zip the front of the suit up to the neck. <p>Doffing:</p> <ol style="list-style-type: none"> 1. Unzip the jumpsuit. 2. Pull the hood off the head. 3. Remove from shoulders and pull arms out of the sleeves. 4. Push jumpsuit down and one at a time life each foot out of the shoe covers of the suit. 5. Properly dispose of the jumpsuit after use.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	N/A
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	Single-use. Dispose of in accordance with facility policy.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A

Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	PPE Brochure, PPE Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com