

## Instructions for Use: Disposable Jumpsuit with Hood and Shoe Covers

<b>Brand Name of Product</b>	Disposable Jumpsuit
<b>Generic Name of Product</b>	Jumpsuit
<b>Product Code Number(s)</b>	30122-M, 3022-L, 30122-XL, 30122-XXL, 30122-XXXL
<b>Intended Use</b>	To cover up person's outer wear from head to toe from coming into contact with items in the decontam area.
<b>Range of Applications for Product</b>	To protect the individual's outer clothing including hair and shoes from contamination.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• White spun-bound material</li> <li>• Elastic cuff and wrist</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Conditions</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use (s)</b>	To keep outer clothes, hair and shoes from coming into contact with contaminated items.
<b>Preparation for Use</b>	<p><b>Donning:</b></p> <ol style="list-style-type: none"> <li>1. Unfold jumpsuit.</li> <li>2. Place one foot into the leg inserting the foot into the shoe cover, then place other foot into the second leg of the shoe cover of the suit.</li> <li>3. Pull jump-suit up and put each arm thru the sleeve of the suit until your hand comes out past the cuff of the sleeve.</li> <li>4. Pull suit up and around the shoulders to cover entire body.</li> <li>5. Place the hood on head to cover the hair entirely.</li> <li>6. Zip the front of the suit up to the neck.</li> </ol> <p><b>Doffing:</b></p> <ol style="list-style-type: none"> <li>1. Unzip the jumpsuit.</li> <li>2. Pull the hood off the head.</li> <li>3. Remove from shoulders and pull arms out of the sleeves.</li> <li>4. Push jumpsuit down and one at a time lift each foot out of the shoe covers of the suit.</li> <li>5. Properly dispose of the jumpsuit after use.</li> </ol>
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	
<b>Interpretation of Test Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	Single-use. Dispose of in accordance with facility policy.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	

<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	PPE Brochure, PPE Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2019-04-16 Suzanne Latta

Rev. A