





















healthmark
INDUSTRIES CO.
health care products




800-521-6224
www.hmark.com

Instructions for Use: Sleeve Gloves

Brand Name of Product	Decontam Long Sleeve Gloves
Generic Name of Product	Sleeve Gloves
Product Code Number(s)	41280, 41380, 41480, 41580, 41680, 41370, 41470, 41570, 41670
Intended Use	For the safe handling of contaminated items in the decontamination area of a sterile processing department. Features long VR™ sleeves connected to gloves for more protection than normal gloves.
Range of Applications for Product	Handling potentially dangerous contaminated items.
Key Specifications of Product	<ul style="list-style-type: none"> • Fusion of 15 mil nitrile latex free lined glove and 4 mil vinyl protective sleeve • 15mm gloves code #: 41280, 41380, 41480, 41580,41680 • 11mm gloves code #: 41370, 41470, 41570, 41670 • Comfortable, flexible and lightweight with elastic cuffing • Available in various lengths and sizes

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product																			
Description of Use(s)																			
Preparation for Use																			
Diagrams (drawings, pictures)	<table border="1"> <tr> <td></td> <td></td> <td></td> </tr> <tr> <td style="text-align: center;">1</td> <td style="text-align: center;">2</td> <td style="text-align: center;">3</td> </tr> <tr> <td colspan="3" style="text-align: center;">DONNING</td> </tr> <tr> <td colspan="3" style="text-align: center;">DOFFING</td> </tr> <tr> <td></td> <td></td> <td></td> </tr> <tr> <td style="text-align: center;">1</td> <td style="text-align: center;">2</td> <td style="text-align: center;">3</td> </tr> </table>				1	2	3	DONNING			DOFFING						1	2	3
																			
1	2	3																	
DONNING																			
DOFFING																			
																			
1	2	3																	

			
Steps for Use of Product	<p>Donning:</p> <ol style="list-style-type: none"> 1. Begin pulling on glove fingers and thumb until they are in the palm of the glove. 2. Pull the fingers of the opposite glove so that both hands are into the palm or cuff area of the gloves. 3. Grasp and pull the fingers of the other glove until the wrist reaches the end of the sleeve – repeat with the opposite hand. <p>Doffing (If reused the same day, it is recommended to be cleaned prior to removal and reused by the same person only):</p> <ol style="list-style-type: none"> 1. Prior to doffing, clean Sleeve Gloves following the cleaning instructions below in the reprocessing section. 2. Begin pulling on glove fingers and thumb until they are in the palm of the glove. 3. Pull the fingers of the opposite glove so that both hands are in the palm or cuff area of the gloves. 4. Grasp and pull the fingers of the other glove until the wrist reaches the end of the sleeve – repeat with the opposite hand. 5. The sleeves are ready for removal once both hands reach the elastic cuffs. 6. Grasp the inside of the opposite sleeve with the thumb and forefinger to remove both sleeve gloves. 7. Sleeve Gloves are ready to hang dry or be properly disposed. If cleaning for reuse, do so prior to doffing. 		
Interpretation of Results			
Contraindications of Test Results			
Documentation			
Special Warnings and Cautions	<ul style="list-style-type: none"> ● Only the same person should wear the same pair of gloves. ● Gloves should be disposed of at the end of each work day of the individual wearing them. ● Prior to donning, carefully inspect the Sleeve Gloves for punctures and cuts. 		
Disposal	Treat used gloves as biohazard and dispose of in accordance with facility policy for contaminated items when disposing.		

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	Clean with mild neutral pH detergent, rinse with clean water. <ul style="list-style-type: none"> ● Wash the gloves while they are being worn (using one glove hand to wash the other). ● Once glove is washed, rinse the gloves with clean water while being worn. ● Once the glove is rinsed, doff as instructed above (if the glove is to be reused it can only be by the same person the same day). ● Do not remove from the decontamination area. ● Hang to dry until next use.
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	

Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	When reusing the gloves (they are only reused by the same person), it is recommended to dispose after use that day (one person, one day).

Related Healthmark Products	
Other Product Support Documents	PPE Instrument Retrieval Brochure, PPE Instrument Retrieval Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-04-19 msmith