




**Instructions for Use: Personal Device Protection Covers**

<b>Brand Name of Product</b>	Personal Device Protection Covers
<b>Generic Name of Product</b>	Disposable Plastic Protector
<b>Product Code Number(s)</b>	68853, 68818, 2005758, SES-002
<b>Purpose of Product</b>	Designed to help protect from moisture, dust and other contaminants.
<b>Range of Applications for Product</b>	For use on your phone, tablet or keyboard.
<b>Key Specifications of Product</b>	Use specific item number designed for the size of the phone, tablet and keyboard. <ul style="list-style-type: none"> <li>• 668853 phone covers, 68818 protective tablet covers, 2005758 Keyboard cover Uni Flex Nano- are outfitted with a zip-lock design.</li> <li>• SES-002- Large phone cover is outfitted with a double zip-lock design.</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Conditions</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Use disposable, zip-lock plastic sleeves to keep your phones and tablets clean. Use keyboard cover to help protect from contaminants.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<p>Phone &amp; Tablet</p> <ol style="list-style-type: none"> <li>1. Slide phone or tablet inside cover.</li> <li>2. Once inside, zip the bag closed and make sure it is secure.</li> </ol> <p>Keyboard and Tablet</p> <ol style="list-style-type: none"> <li>1. Take the cover out of package.</li> <li>2. Cover the entire keyboard with protection cover.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	Damaged covers should be replaced immediately. Yellow may be an indicator that the cover is expired.
<b>Disposal</b>	Dispose as needed in accordance with your facility's hazardous materials disposal policy.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	Cleaning should be performed on keyboard only when cover is completely pulled over the keyboard. Take a slightly moist cloth using warm water and facility approved cleaning solution, or 70% Isopropyl Alcohol, and wipe cover thoroughly. Do not submerge in water or wipe cover from the inside as humidity will occur.
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	SST Brochure, SST Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2017-11-10 Matt Smith