



**Instructions for Use: STA-DRI Disposable Jumpsuit**

<b>Brand Name of Product</b>	STA-DRI Disposable Jumpsuit
<b>Generic Name of Product</b>	Disposable Jumpsuit
<b>Product Code Number(s)</b>	STA-DRI
<b>Intended Use</b>	To help keep technician dry when reprocessing dirty instruments in the decontamination area.
<b>Range of Applications for Product</b>	To protect and keep you dry from organic soil and water splash up.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>● Non-skid disposable jumpsuit.</li> <li>● One size fits all.</li> <li>● Made of impermeable polyethylene for use with gown.</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Conditions</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	To keep technicians protected from splashing and protect against organic soils.
<b>Preparation for Use</b>	Remove jumpsuit from packaging prior to use.
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<p><b>Donning:</b></p> <ol style="list-style-type: none"> <li>1. Unfold jumpsuit.</li> <li>2. Place one foot into the leg down into the boot section of the jumpsuit.</li> <li>3. Then place the other foot into the second leg of the jumpsuit into the boot section.</li> <li>4. Make sure your feet are securely placed in the boot of the jumpsuit.</li> <li>5. Pull the jumpsuit up the legs and body and pull the neck strap over the head to rest on the back of the neck.</li> </ol> <p><b>Doffing:</b></p> <ol style="list-style-type: none"> <li>1. With gloves still on, pull neck strap back over head.</li> <li>2. Pull jumpsuit down to feet and remove feet one at a time from boot cover.</li> <li>3. Properly dispose of jumpsuit.</li> </ol>
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	Single-use, treat as biohazard. Dispose of in accordance with facility policy for contaminated items.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	

<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	PPE Brochure, PPE Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 33671 Doreka Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2017-11-17 Suzanne Latta