


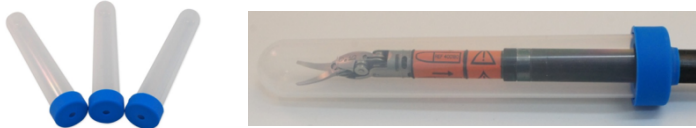

**Instructions for Use: Distal Soak**

|  |  |
|--|--|
| <b>Brand Name of Product</b>             | Distal Soak  |
| <b>Generic Name of Product</b>           | Maintaining a moist state and protecting tips during transport   |
| <b>Product Code Number(s)</b>            | 50058, 50065, 50069  |
| <b>Intended Use</b>                      | The Distal Soak is used during the post-procedure transport. The set keeps the distal end moist and facilitates reprocessing, preventing residuals from drying. Furthermore, the delicate tips are protected during transport. |
| <b>Range of Applications for Product</b> | Distal tips  |
| <b>Key Specifications of Product</b>     | <ul style="list-style-type: none"> <li>• Polypropylene (PP) tubes</li> <li>• Silicone lids</li> </ul>  |

| <b>Shipping &amp; Storage</b>                 |  |
|---|--|
| <b>Shipping Conditions &amp; Requirements</b> |  |
| <b>Storage Conditions</b>                     |  |
| <b>Packaging Conditions</b>                   |  |
| <b>Shelf Life</b>                             |  |

| <b>Instructions for Using Product</b>    |   |
|--|---|
| <b>Description of Use(s)</b>             | <ul style="list-style-type: none"> <li>• Keeps the distal end moist and facilitates reprocessing.</li> </ul>  |
| <b>Diagrams (drawings, pictures)</b>     |    |
| <b>Steps for Use of Product</b>          | <p><b>Closed Hole Distal Soak</b></p> <ol style="list-style-type: none"> <li>1. After the procedure, take the Distal Soak tube out of the packaging unit.</li> <li>2. Fill the tube with a solution that complies with the item's IFU.</li> <li>3. Push the distal end through the closed hole in the cap.</li> <li>4. Transport the device to the processing area.</li> <li>5. When ready to begin further reprocessing, remove the from the Distal Soak.</li> </ol> <p><b>Open Hole Distal Soak</b></p> <ol style="list-style-type: none"> <li>1. After the procedure, take the open hole Distal Soak tube out of the packaging unit.</li> <li>2. Push the distal end of the instrument through the open hole into the empty tube.</li> <li>3. Fill the tube with a solution that complies with the item's IFU. Maximum volume of the tube is 15 mL.</li> <li>4. Transport the device to the processing area.</li> <li>5. When ready to begin further reprocessing, remove from the Distal Soak.</li> </ol> |
| <b>Interpretation of Results</b>         |   |
| <b>Contraindications of Test Results</b> |   |
| <b>Documentation</b>                     |   |
| <b>Special Warnings and Cautions</b>     | <ul style="list-style-type: none"> <li>• Consult the item manufacturer's IFU for proper precleaning procedures.</li> <li>• Where appropriate PPE in compliance with facility's policy.</li> </ul>   |
| <b>Disposal</b>                          | <ul style="list-style-type: none"> <li>• Single use</li> <li>• Treat used Distal Soak as a biohazard and dispose of in accordance with facility policy for contaminated items.</li> </ul>   |

| <b>Reprocessing Instructions</b>       |  |
|--|--|
| <b>Point of Use</b>                    |  |
| <b>Preparation for Decontamination</b> |  |
| <b>Disassembly Instructions</b>        |  |
| <b>Cleaning – Manual</b>               |  |
| <b>Cleaning – Automated</b>            |  |
| <b>Disinfection</b>                    |  |

|   |  |
|---|--|
| <b>Drying</b>                               |  |
| <b>Maintenance, Inspection, and Testing</b> |  |
| <b>Reassembly Instructions</b>              |  |
| <b>Packaging</b>                            |  |
| <b>Sterilization</b>                        |  |
| <b>Storage</b>                              |  |
| <b>Additional Information</b>               |  |

|  |  |
|--|--|
| <b>Related Healthmark Products</b>     |  |
| <b>Other Product Support Documents</b> | ProSys™ Brochure, ProSys™ Price List   |
| <b>Reference Documents</b>             |  |
| <b>Customer Service Contact</b>        | Healthmark Industries Company, Inc.<br>18600 Malyn Blvd.<br>Fraser, MI 48026<br>1-586-774-7600<br><a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a><br><a href="http://hmark.com">hmark.com</a> |