





Instructions for Use: SST Liners

Brand Name of Product	SST Liners
Generic Name of Product	SST Liners
Product Code Number(s)	SST-LNR-CL, SST-LNR-RD, SST-LNR-GN, SST-CCD-LNR
Intended Use:	To line the bottom of Round Soakers and other containers to protect from staff and patients from contaminated instruments during transportation. Also provides clear messaging on equipment being clean or dirty.
Range of Applications for Product	For use in endo bins and containers.
Key specifications of product	<ul style="list-style-type: none"> • Single-use • Green bags state “Clean” messaging • Red bags state “Biohazard” messaging

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	To line the bottom of Round Soakers and other containers to protect from staff and patients from contaminated instruments during transportation. Also provides clear messaging on equipment being clean or dirty.
Preparation for Use	Container used must be cleaned and reprocessed according to manufactures IFU before using.

Diagrams (drawings, pictures):			
	1	2	3
			
	4	5	6

Steps for Use of Product	<p>Traditional Use:</p> <ol style="list-style-type: none"> 1. Place clear liner into the empty container. 2. Place items in the container. 3. Place folded red liner with the scope. 4. The green “Clean” liner covers the scope. 5. Once equipment is used, put in back in the container with the clear liner and place the red “Biohazard” liner over the item used. 6. Place lid on the container.
---------------------------------	--

	Other Variations of Use: All three liners can be used individually as single use liners.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	Single-use. Dispose of in compliance with facility policy for disposing of all biohazard items.

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	
Storage:	
Additional Information:	

Related Healthmark Products	
Other Product Support Documents	PPE Brochure, PPE Price List
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2017-01-09 msmith