



**Instructions for Use: SST Tray Stands**

<b>Brand Name of Product</b>	SST Tray Stands
<b>Generic Name of Product</b>	SST Tray Stands
<b>Product Code Number(s)</b>	2136-STAND, 2136-STAND-2
<b>Intended Use</b>	To transport SST Trays that contain contaminated instruments.
<b>Range of Applications for Product</b>	Transporting SST Trays to the decontam room.
<b>Key Specifications of Product</b>	<ol style="list-style-type: none"> <li>1. Made of 100% stainless steel.</li> <li>2. Comes in 2 different sizes: <ul style="list-style-type: none"> <li>• 15.75in.L x 28.50in.W x 33in.H</li> <li>• 22in.L x 34in.W x 36in.H</li> </ul> </li> <li>3. Comes fully assembled.</li> <li>4. Single tray or double tray style.</li> </ol>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	Comes fully assembled
<b>Storage Conditions</b>	
<b>Packaging Conditions</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use (s)</b>	To hold SST Trays that contain contaminated instruments and also used to transport clean trays after stand has been cleaned and disinfected.
<b>Preparation for Use</b>	Remove stand from shipping box.
<b>Diagrams (drawings, pictures):</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. With gloved hands, put SST Tray(s) into the rings of the stand.</li> <li>2. Place one hand on each of the rounded corners of the stand and transport to the decontamination room.</li> </ol>
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	To clean, place SST Tray Stand in cart washer.
<b>Disinfection</b>	The thermal disinfection stage of the cart washer may be used to disinfect the transport cart.
<b>Drying</b>	After cart washer cycle is completed, let stand dry and cool down before touching.
<b>Maintenance, Inspection, and Testing</b>	Inspect the wheels for maintenance to be sure they do not come loose.
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	SST Accessories

<b>Other Product Support Documents</b>	PPE Brochure, PPE Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2017-12-04 Suzanne Latta