

**Instructions for Use: SST Transport Carts**

<b>Brand Name of Product</b>	SST Transports Carts
<b>Generic Name of Product</b>	Instrument retrieval carts
<b>Product Code Number(s)</b>	113-20, 113-1220, 113-30, 113-40, 113-40, 113-50, TTCC-20, TTC-40, 300DIV.
<b>Purpose of Product</b>	To safely transport and store SST instrument retrieval trays in water-tight closed carts.
<b>Range of Applications for Product</b>	Small, mid size or large carts for SST transportation.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Manufactured of tough polyethylene.</li> <li>• Double wall construction.</li> <li>• Swivel and locking casters.</li> <li>• Water tight seal when doors latched shut.</li> <li>• Safely and securely transport SST trays.</li> <li>• Carts are divided into two compartments (except TTCC-20 which is one compartment) allowing for delivery of “clean” and pick-up of “dirty” containers at the same time.</li> <li>• Stainless steel casters with sealed bearings.</li> <li>• Completely cart washable.</li> </ul>

**Shipping & Storage**

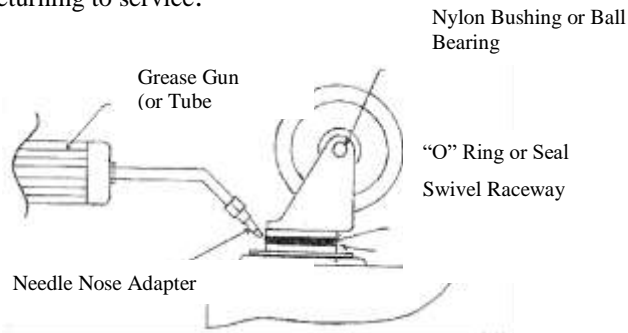
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Conditions</b>	
<b>Shelf Life</b>	

**Instructions for Using Product**

<b>Description of Use (s)</b>	For transportation of SST instrument retrieval trays.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ul style="list-style-type: none"> <li>• Place “Clean” SST Trays inside for delivery to the OR or other clinical area.</li> <li>• Pick up “Dirty” Containers at the same time to deliver to Decontam.</li> </ul>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	<p>If designating sections as “clean” and “dirty, never place clean trays in the section designated for dirty trays and vice versa.</p> <p>If sections are not segregated as “clean” and “dirty”, following the reprocessing instructions below thoroughly clean and disinfect after transportation of dirty trays.</p>
<b>Disposal</b>	

**Reprocessing Instructions**

<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	<ul style="list-style-type: none"> <li>• May be cleaned with a steam gun or</li> <li>• Scrubbed with a lint free towel, sponge or nylon bristle brush (never use a metal bristle brush) using a cleaning solution comprised of a non-abrasive, neutral pH detergent and water at the temperature recommended in the detergent manufacturer’s IFU.</li> <li>• Rinse thoroughly with clean water.</li> </ul>
<b>Cleaning – Automated</b>	<ul style="list-style-type: none"> <li>• All SST carts are cart washer safe.</li> <li>• Doors should be open.</li> <li>• Avoid latches coming in contact with inside walls of the washer.</li> <li>• Be sure to consult and follow IFU from the washer manufacturer.</li> <li>• Cycle Settings:             <ol style="list-style-type: none"> <li>1. 2 minute rinse with water from the facility cold water source</li> <li>2. Detergent wash with neutral pH detergent. Be sure to follow the IFU from the detergent manufacturer.</li> </ol> </li> </ul>

	3. Thermal disinfection rinse at 180°F for 1 minute.
<b>Disinfection</b>	Thermal disinfection stage of the cart washer. Maximum time & temperature is 180°F.
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	<p>Inspect all gaskets after cleaning to make sure they are intact (no cracks) and replace as needed.</p> <p><u>MAINTENANCE OF CASTERS</u></p> <p>Maintenance care of casters, <i>each time</i> the unit has been washed or steam cleaned, needs to be as follows:</p> <ol style="list-style-type: none"> <li>Grease swivel raceway of caster using a lubricant such as: Lubriplate No. 930-AA Keystone #84EPXLT Multi-Purpose Grease or equivalent</li> </ol> <p>Moving “O” ring or seal with finger, apply grease using a needle nose adapter on grease gun or a tube type applicator such as “Sta-lube” Moly-graph Assembly Lube. These can be obtained at most auto parts stores.</p> <ol style="list-style-type: none"> <li>On casters [swivel or stationary] with a bearing hub wheel no lubrication is needed. On casters [swivel or stationary] using a nylon bushing at the hub, use light oil such as WD-40.</li> <li>When done, check caster movement both in wheel spin and swivel rotation for any damage before returning to service.</li> </ol> 
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	SST Trays
<b>Other Product Support Documents</b>	SST Brochure, SST Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	<p>Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a></p>