



33671 Doreka  
Fraser, Michigan 48026  
Telephone: (586) 774-7600  
Fax: (586) 774-6473  
[www.HMARK.com](http://www.HMARK.com)

## Sealed Transport Cart Cleaning Instructions

- All Sealed Transport carts are cart washer safe (neutral pH detergent). They can also be cleaned by steam (steam gun) or washed manually with warm soapy water and a plastic or a bristle brush. (Never use a metal brush or abrasive cleaners).
  - If manually cleaned, scrub with brush, wash down and rinse thoroughly with a clean source of water.
- If the Sealed Transport carts go through a cart washer, be sure that the doors are open and latched do not get caught in any of the spray attachments.
- Inspect all gaskets after cleaning to make sure they are intact (no cracks) replace as needed..