



33671 Doreka
Fraser, Michigan 48026
Telephone: (586) 774-7600
Fax: (586) 774-6473
www.HMARK.com

Clear-Top Micro Protech Trays

0278, 02781, 027814, 0625, 062514, 6540, 7540, 754015
100675, 100615, 100615B, 101525, 261615, 302215
Plus all silicone accessories (Mats & Plug-in Secur-its)

PST instrument trays are designed to encase instruments during processing and are safe for all standard sterilization methods: Autoclave/dry heat up to 360°F, ETO, and Cold Solutions.

- Tray will last over 1000 cycles when properly handled and processed.
- An appropriate sterile barrier will be required to maintain sterility.
- Use of the PST Instrument Trays must be in accordance with the operating instruction supplied with your particular sterilization equipment and in keeping with your particular sterilization equipment and in keeping with your hospital policy on sterilization validation. Several organizations (e.g., AAMI, AORN and ISO) publish guidance information on sterilization usage should you need additional information.
- Periodic cleaning of the trays after sterilization process is recommended to prolong usable life.
- Do not use if damaged

PST
(Plastic Sterilizing Tray Corporation)
Pittsfield, MA USA 01201

Distributed by: Healthmark Industries Co. 33671 Doreka Drive, Fraser MI 48026
Phone 800-521-6224 Fax 586-491-2112