To OPEN container -  
firmly press center front of  
container below lid and  
between latches.

NOTE: Remove sticker before use.

**CAUTION:**

- Do not dry heat sterilize
- Do not allow container to touch side walls of sterilizer.
- Do not exceed temperature of 270 degrees Fahrenheit

**ZIRC COMPANY**

3918 Hwy 55 SE Buffalo, MN 55313 800-328-3899 zirc.com  
Authorized Rep acc. to MDD 93/42 EEC LOSER & CO.  
Benzstr. 1c D51381 Leverkusen Germany +49-2171-706670

Zirc Company  
Processing of Reusable Medical Devices  
According to EN ISO 17664:2004

CE

Manufacturer: Zirc Company, 3918 Hwy 55 SE, Buffalo, MN 55313 USA  
Products: Mini-Matt, Steri-Container, Compact Steri-Container, Steri-Instrument Cassette, Compact Cassette & Components

CE (REP) Loser & Co. GmbH Benzstr. 1-3, D51381 Leverkusen Germany +49-2171-706670

NOTE: Please follow the instructions for use from the manufacturer of your cleaning and sterilization equipment. Hereinafter called "see manufacturer information".

1. Restriction of Reprocessing: Frequent reprocessing has marginal consequences for Cassettes & Containers. Normally the end of durability of the product will be determined by visual degradation.
2. Preparation Before Cleaning: Place contaminated instruments/medical devices in the Steri-Cassettes/Containers and ensure the items are secure. Please note: For the Compact Steri-Container a plastic "key" is needed for the initial disassembly (see instructions for use).
3. Cleaning: Place the Cassettes/Containers in the Ultrasonic / Automated Washer (see manufacturer information) then rinse thoroughly.
4. Packaging: Bag or wrap Cassettes/Containers if necessary.
5. Sterilization: The Cassettes/Containers are intended to hold your instruments/medical devices (follow the sterilization process determined by the instrument/medical device manufacturer). Please note: The Cassettes/Containers are sterilizable up to 136°C. And are not "dry-heat" sterilizable. 20234017 Rev. 8/03

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