



33671 Doreka  
Fraser, Michigan 48026  
Telephone: (586) 774-7600  
Fax: (586) 774-6473  
www.HMARK.com

**Sterilization Processing Instructions  
Hospital Instrument Case/Tray Systems  
Manufactured by PST**

0278, 02781, 027814, 0625, 062514, 100615,  
100615B, 100675, 101525, 101525B, 6540, 7540 & 754015

PST instrument trays are designed to encase instruments during processing and are safe for all standard sterilization methods: Autoclave/dry heat up to 360°F, ETO, and Cold Solutions.

- Tray will last over 1000 cycles when properly handled and processed.
- An appropriate sterile barrier will be required to maintain sterility.
- Use of the PST Instrument Trays must be in accordance with the operating instruction supplied with your particular sterilization equipment and in keeping with your particular sterilization equipment and in keeping with your hospital policy on sterilization validation. Several organizations (e.g., AAMI, AORN and ISO) publish guidance information on sterilization usage should you need additional information.
- Periodic cleaning of the trays after sterilization process is recommended to prolong usable life.
- Do not use if damaged