

Brand Name of Product	Scope Mesh Container
Generic Name of Product	Scope Mesh Container
Product Code Number(s)	SMT5510
Purpose of Product	Rigid mesh container with lid and silicone grips included are intended for the use of securely transporting scopes and long instruments during decontamination, washing, disinfection and sterilization process.
Range of Applications for Product	Scopes and long instruments
Key Specifications of Product	<ul style="list-style-type: none"> • Scope Mesh Container • Scope Mesh Lid • Silicone Grips x2 • Two Silicone Plugs

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	<ul style="list-style-type: none"> • Must not be handled roughly or dropped
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Place scopes or instruments in mesh container when during reprocessing
Preparation for Use	Remove Silicone Grips from bag to insert in the container
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Position Silicone Grip above any of the location slots 2. Push down to secure Grip in place (To remove, hold Silicone Grip bottom corner and pull upwards)

	<ol style="list-style-type: none"> 3. After instruments are placed securely between the grips in the container, position lid so pips insert into holes and push down 4. To secure the plug, pull the bottom of the plug through both holes until the dimple appears. 5. When removing the plug, hold the top of the silicone plug, squeeze and gently pull upwards.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	<ul style="list-style-type: none"> • Do not place heavy weight on top of container
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	See Protech Trays IFU for instructions
Cleaning – Automated	See Protech Trays IFU for instructions
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	See Protech Trays IFU for instructions
Storage	
Additional Information	

Related Healthmark Products	ProTech™ Brochure, ProTech™ Price List
Other Product Support Documents	Protech Trays IFU
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn BLVD. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2019-04-23 Suzanne Latta