

Instructions for Use: Magnetic Mats

Brand Name of Product	Magnetic Mats
Generic Name of Product	Mats
Product Code Number(s)	11116
Intended Use	Magnetic Mats provide a magnetized non-slip surface that minimizes accidental injuries and droppage by providing an alternative to the hand-to-hand transfer.
Range of Applications for Product	Hands-free transfer
Key Specifications of Product	<ul style="list-style-type: none"> • Silicone Rubber, magnetized non-slip surface • Color: Purple • 16" x 20" • Autoclavable • Special cutting lines for cutting the mat to 2 smaller mats as necessary for smaller procedures • Waterproof

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	Store and handle with care.
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use(s)	Magnetic Mats provide a magnetized non-slip surface that minimizes accidental injuries and droppage by providing an alternative to the hand-to-hand transfer.
Preparation for Use	
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Lie flat or secure the magnetic mat through the holes at the corners on an uneven surface. 2. Do not perforate any other location on the magnetic mat. 3. Place item on magnetic mat when not in use or when non hand-to-hand transfer is desired.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	<ul style="list-style-type: none"> • Do not place on or near patient with pacemaker or similar device without first checking compatibility with such devices. • Avoid contact with the following substances: <ul style="list-style-type: none"> ○ Oils, ointment and oil containing soaps (green soaps) ○ Hydrocarbon solvents: ether, acetone and hydrochloric acids, lye and sodium hypochlorite (bleach) ○ Phenol and hexachlorophene products ○ Quaternary compounds and benzalkonium chloride
Disposal	This item is to be disposed as biohazardous waste in compliance with the healthcare facility's policy

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual:	Wash magnetic mat with mild detergent. Rinse with water and dry with non-linting material.
Cleaning – Automated	
Disinfection	
Drying:	

Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	<ul style="list-style-type: none"> • The magnetic mat should be sterilized flat without folding and should not be placed on a magnetic metallic surface during steam sterilization. • Recommended cycle time for steam sterilization is 132°C for 4 minutes, 135°C for 3 minutes. • Gravity cycle at 121°C for 30 minutes.
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	ProTech® Brochure, ProTech® Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2020-08-06 Matt Smith