











Instructions for Use: Handheld labeling System



Brand Name of Product	Handheld Labeling System
Generic Name of Product	Label Gun and labels
Product Code Number(s)	H-362026, H362026, H369021, H369023, H369024, H369024, H369010, H369012, H369013, H369014, H369020S, H369020S, H369021S, H369023S, H369024S, H369010S, H369012S, H369013S, H369014S
Purpose of Product	Hand-held Labeling System has been designed specifically with Central Sterile in mind for the labeling of any items that you need to be autoclaved - from sterile linen packs to instrument trays, and peel pouches.
Range of Applications for Product	Great versatility for labeling in other hospital departments such as dietary and pharmacy.
Key specifications of product	All of the labels are compatible with Steam and EO sterilization. The labels with an “S” at the end of the product code number are compatible with Gas-Plasma Sterilization.

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Label Gun is used for labeling a variety of products and packages being autoclaved.		
Preparation for Use			
Diagrams (drawings, pictures):	<p>Setting the characters</p> <div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>1</p> </div> <div style="text-align: center;">  <p>2</p> </div> </div> <p>Labeling</p> <div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>1</p> </div> <div style="text-align: center;">  <p>2</p> </div> <div style="text-align: center;">  <p>3</p> </div> <div style="text-align: center;">  <p>4</p> </div> </div> <p>Replacing the label roll</p> <div style="display: flex; flex-wrap: wrap; justify-content: space-around;"> <div style="text-align: center; width: 30%;">  <p>1</p> </div> <div style="text-align: center; width: 30%;">  <p>2</p> </div> <div style="text-align: center; width: 30%;">  <p>3</p> </div> <div style="text-align: center; width: 30%;">  <p>4</p> </div> <div style="text-align: center; width: 30%;">  <p>5</p> </div> </div>		

	<p>Replacing the ink-roll</p> <div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>1</p> </div> <div style="text-align: center;">  <p>2</p> </div> </div>
--	---

Steps for Use of Product	<p>Setting the characters</p> <ol style="list-style-type: none"> 1. Pull adjustment knob to desired position and turn knob to select character to print. 2. Press knob in before printing (Important: failure to do so before operating could cause damage). <p>Labeling</p> <ol style="list-style-type: none"> 1. Place your hand around the handle and squeeze you fingers to your palm. 2. Release the handle back to the starting position. 3. Pulling the handle will release the label from the opposite end of the label gun. 4. Repeat to use another label on another product or package. <p>Replacing the label-roll</p> <ol style="list-style-type: none"> 1. Press and slide red release for the top slide back. The release is on both sides of the label gun. 2. Pull out the chamber in order to insert the roll of labels. 3. Place new label roll inside with the label flap coming over the top. 4. Secure the end of the label roll to the top end of the label gun. 5. When in use, the label will dispense through the chamber. <p>Replacing the ink-roll</p> <ol style="list-style-type: none"> 1. Press and slide red release of the label gun. There is a release on both sides. 2. Raise the ink roll and pull outward. (The side notches keep the ink roll in place. Pushing the back end of the ink roll downward will also release it. We recommend wearing gloves. Your hands will get ink on them.) Grab a new ink roll. Place the new in roll into the label gun (Sided notched will click in place when secured).
---------------------------------	---

Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	Use damp cloth to wipe down the Label Gun.
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	All of the labels are compatible with Steam and EO sterilization. The labels with an “S” at the end of the product code number are compatible with Gas-Plasma Sterilization. (DO NOT PLACE THE LABEL GUN IN ANY STERILIZER).
Storage:	
Additional Information:	

Related Healthmark Products	Premium Label Gun and two-part labels
Other Product Support Documents	Labeling Brochure, Labeling Price List
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2014-05-28 Ralph J Basile