

Instructions for Use: Quick Turnover 2-Part Clean-Dirty Label

Brand Name of Product	Quick Turnover 2-Part Clean-Dirty Label
Generic Name of Product	Tray tag
Product Code Number(s)	HM-52489
Intended Use	To identify the status of items during the reprocessing cycle. Label contains pull tags that indicate if items are clean or dirty and/or items needed for quick turnover.
Range of Applications for Product	Case carts or anything used to transport items during reprocessing.
Key Specifications of Product	<ul style="list-style-type: none"> ● 3.125" x 5.125" Perforated Tags ● Adhesive Backing ● Two Tags: <ul style="list-style-type: none"> - Top Tag <ul style="list-style-type: none"> ● Transportation Tag, Quick Turnover Tag - Bottom Tag <ul style="list-style-type: none"> ● Clean Tag, Dirty Biohazard Tag

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	Keep in dry, room temperature area.
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use(s)	The Quick Turnover Label can be used on any piece of equipment/cart that is assigned to transport supplies with the potential to return containing contaminated items and/or for quick turnover.
Preparation for Use	Should be applied to a clean, dry surface.
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Removing label from the plastic backing: <ul style="list-style-type: none"> ● Before placing the label on the transportation cart/bin, slightly bend the bottom ½" of the label on the perforation. This will assist in the removal of the label in decontamination by creating a free piece of the label to grab. 2. Placing the label on the cart/bin: <ul style="list-style-type: none"> ● The label should be placed in a visual place for staff to see. This is typically on the door or top of the transportation cart/bin. 3. After the procedure: <ul style="list-style-type: none"> ● Hands should be covered with a clean pair of gloves for removal of the sticker. ● Remove the TRANSPORTATION TAG by pulling up where it says "Pull Here" in a right to left fashion to reveal the QUICK TURNOVER label underneath. ● Write what items are needed for quick turnover and the time needed before returning to decontamination for reprocessing. ● Remove the CLEAN tab by pulling up where it says "Pull Here" in a right to left fashion to reveal the DIRTY BIOHAZARD label underneath. 4. Removing the label in decontamination: <ul style="list-style-type: none"> ● Prior to removing the label, read and identify what items are needed for quick turnover. ● Reprocess the items needed in order to be sterilized before said time on the Quick Turnover Label. ● Clean all items per your facility policy and empty the transportation cart/bin. 5. Removing the label in decontamination:

	<ul style="list-style-type: none"> Remove the label by grabbing the bottom area just below the perforation. Pull in a slow motion up and away from the cart. DO NOT PULL FAST. Residue may be left behind if pulled too quickly. Label can be discarded in the garbage.
Interpretation of Test Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	Do not wash or sterilize with instruments.
Disposal	The Quick Turnover Label is a one-time single use label and is to be discarded after each use in a facility approved waste container according to policy.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	
Related Healthmark Products	Transportation Identification Label
Other Product Support Documents	Labeling Brochure, Labeling Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-11-29 Suzanne Latta