



Instructions for Use: Self-Looping labels IFU

Brand Name of Product	Self-Looping Labels
Generic Name of Product	Tray labels
Product Code Number(s)	403005, 403115, 403125, 403225, 404225-351-GI, 403225 LOANER, 403225 PINK, 403225 RED, 403225 RED BIO, 403225 SLOTTED, 403225 SLOTTED PINK, 403225 YELLOW, 403225 BLUE, 403225 GREEN, TAG-CONTAIN2
Purpose of Product	These labels facilitate specific markings and remarks for the sterile contaminate trays. They are designed to minimize misunderstandings about the contents of an instrument tray and to notify about maintenance of an instrument.
Range of Applications for Product	
Key specifications of product	<ul style="list-style-type: none"> • Self-looping style for mounting on the handle of any closed container • Compatible with steam, EO and VHP sterilization • Available in a variety of colors

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Conditions	Packaged 500 per box
Shelf Life	

Instructions for Using Product	
Description of Use (s)	The PET- based label is suitable for sterilization and are resistant to washing and disinfection and thus can go all the way through reprocessing.
Preparation for Use	
Diagrams (drawings, pictures):	
Steps for Use of Product	<ol style="list-style-type: none"> 1. You will loop the tray label through the handle of the container or tray. 2. When using slotted tray label, push the count sheet through the tray label so it is hanging evenly. You can also affix an ID label to the front of the tag.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	<ul style="list-style-type: none"> • Tray labels should not be in physical contact with surgical instruments during sterilization cycles. • Tray labels should not be blocking any vent holes in trays as this may interfere with sterilization. • Do not handle tray labels until they are allowed to cool down to room temperature after sterilization or thermal disinfection.
Disposal	

Reprocessing Instructions	
Point of use:	
Preparation for decontamination:	
Disassembly Instructions:	
Cleaning – Manual:	
Cleaning – Automated:	
Disinfection:	
Drying:	
Maintenance, inspection, and testing:	
Reassembly Instructions:	
Packaging:	
Sterilization:	<ul style="list-style-type: none"> • Compatible with steam, EO and VHP sterilization. • Recommended cycle time for steam sterilization is 270°F for 4 minutes / 275°F for 3 minutes.
Storage:	
Additional Information:	Make sure label is hanging freely from the tray handle and not blocking holes in tray.

Related Healthmark Products	
Other Product Support Documents	Labels Brochure, Labels Price List
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2017-08-17 msmith