

Slotted tray label Instructions for Use

The slotted tray label is a product that allows you to attach your surgical instrument count sheet on the outside of a container. It eliminates the concern of ink transfer from the count sheet to any instrument.



The slotted tray label has two slots to hold the rolled up count sheet.



After you have secured (closed, locked) the container; you will loop the slotted tray tag through the handle of the container.



Take the surgical count sheet and roll it (when possible with the ink on the inside of the roll) and then place it between the two slots.



Push the count sheet through the slotted tray label so it is evenly hanging on the slotted tray label. You can also affix an ID label to the front of the tag.