









**Instructions for Use: Transportation Identification Label**

|  |   |
|--|---|
| <b>Brand Name of Product</b>             | Transportation Identification Label   |
| <b>Generic Name of Product</b>           | Tray Tags   |
| <b>Product Code Number(s)</b>            | AV-52481, AV-52481-RL, AV-52482, AV-52482-RL, HM-CID-613, HM-52483-HLD  |
| <b>Intended Use</b>                      | To identify the status of instruments during the reprocessing cycle. Label contains pull tabs that indicate if instruments are clean or dirty. They are single use.                             |
| <b>Range of Applications for Product</b> | Case carts or anything used to transport surgical instruments during reprocessing.  |
| <b>Key Specifications of Product</b>     | <ul style="list-style-type: none"> <li>• 3.125" x 5.125" perforated tabs</li> <li>• Adhesive backing</li> <li>• Available with and without checklist towards the bottom of the label</li> </ul> |

| <b>Shipping &amp; Storage</b>                 |                                     |
|---|-------------------------------------|
| <b>Shipping Conditions &amp; Requirements</b> |                                     |
| <b>Storage Conditions</b>                     | Keep in dry, room temperature area. |
| <b>Packaging Conditions</b>                   |                                     |
| <b>Shelf Life</b>                             |                                     |

| <b>Instructions for Using Product</b>   |  |   |  |     |     |
|---|--|---|--|-----|-----|
| <b>Description of Use(s)</b>  | The Transportation Label can be used on any piece of equipment/cart that is assigned to transport supplies, with the potential to return containing contaminated items.  |   |  |     |     |
| <b>Preparation for Use</b>  | Should be applied to a clean, dry surface.   |   |  |     |     |
| <b>Diagrams (drawings, pictures):</b>   | <table border="1" style="width: 100%; text-align: center;"> <tr> <td></td> <td></td> </tr> <tr> <td>1-3</td> <td>4-5</td> </tr> </table>  |  |  | 1-3 | 4-5 |
|  |    |   |  |     |     |
| 1-3   | 4-5  |   |  |     |     |
| <b>Steps for Use of Product</b>   | <ol style="list-style-type: none"> <li>1. Filling out information on Transportation Label ( not all labels have space for information)<br/>           Upon completion of assembling your transportation cart/bin, fill out the top portion of the label.<br/>           Using a marker fill out the following ( if applicable ) :           <ul style="list-style-type: none"> <li>○ Date (When does the procedure take place?)</li> <li>○ Case/Procedure (Intended use for the contents)</li> <li>○ Room/Facility (Where is the procedure taking place?)</li> <li>○ Initials (Who picked/assembled the supplies?)</li> </ul> </li> <li>2. Removing label from the plastic backing<br/>           Before placing the label on the transportation cart/bin, slightly bend the bottom ½" of the label on the perforation. This will assist in the removal of the label in decontamination by creating a free piece of the label to grab.</li> <li>3. Placing the label on the cart/bin<br/><br/>           The label should be placed in a visual place for staff to see. This is typically on the door or top of the transportation cart/bin.</li> <li>4. After the procedure           <ul style="list-style-type: none"> <li>○ The staff fills out the transportation checklist on the bottom portion of the label to designate all safety checks have been complete, including staff initials and OR room/facility. This includes marking yes, no or N/A for the following:               <ul style="list-style-type: none"> <li>• Linen has been removed</li> </ul> </li> </ul> </li> </ol> |   |  |     |     |

|  |  |
|--|--|
|  | <ul style="list-style-type: none"> <li>● All sharps have been removed</li> <li>● Instruments are pre-cleaned</li> <li>● Clean and dirty instruments are separated</li> <li>● Initials and room #</li> </ul> <p>* The staff in decontamination must be able to trace the contaminated instruments back to a specific location of use or patient in the unfortunate event of a bloodborne pathogen exposure.</p> <p>5. Removing the clean tab on the label</p> <ul style="list-style-type: none"> <li>● Hands should be covered with a clean pair of gloves for removal of the sticker</li> <li>● Remove the “Clean” tab by pulling up where it says “Pull Here” in a right to left fashion.</li> <li>● The biohazard symbol and the word “Dirty” will be revealed under the “Clean” tab once it is removed.</li> </ul> <ul style="list-style-type: none"> <li>▪ Removing the label in decontamination <ul style="list-style-type: none"> <li>● Prior to removing the label, clean all the instruments per your facility policy and empty the transportation cart/bin.</li> <li>● Remove the label by grabbing the bottom area just below the perforation. Pull in a slow motion up and away from the cart. DO NOT PULL FAST. Residue may left if pulled too quickly.</li> <li>● Label can be discarded in the garbage.</li> </ul> </li> </ul> |
| <b>Interpretation of Results</b>         |  |
| <b>Contraindications of Test Results</b> |  |
| <b>Documentation</b>                     |  |
| <b>Special Warnings and Cautions</b>     | Do not wash or sterilize with instruments.   |
| <b>Disposal</b>                          | The Transportation Label is a one-time use label and must be discarded after each use in a facility approved waste container according to policy.  |

| <b>Reprocessing Instructions</b>           |  |
|--|--|
| <b>Point of Use</b>                        |  |
| <b>Preparation for Decontamination</b>     |  |
| <b>Disassembly Instructions</b>            |  |
| <b>Cleaning – Manual</b>                   |  |
| <b>Cleaning – Automated</b>                |  |
| <b>Disinfection</b>                        |  |
| <b>Drying</b>                              |  |
| <b>Maintenance, Inspection and Testing</b> |  |
| <b>Reassembly Instructions</b>             |  |
| <b>Packaging</b>                           |  |
| <b>Sterilization</b>                       |  |
| <b>Storage</b>                             |  |
| <b>Additional Information</b>              |  |

|  |  |
|--|--|
| <b>Related Healthmark Products</b>     |  |
| <b>Other Product Support Documents</b> | Labeling Brochure, Labeling Price List   |
| <b>Reference Documents</b>             |  |
| <b>Customer Service Contact</b>        | Healthmark Industries Company, Inc.<br>33671 Doreka<br>Fraser, MI 48026<br>1-586-774-7600<br><a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a><br>hmark.com |