


Instructions for Use: Tray Identification Tags

<b>Brand Name of Product</b>	Tray Identification Tags
<b>Generic Name of Product</b>	Tray Identification Tags
<b>Product Code Number(s)</b>	RIN001, TAG001, TAG011, TAG012, TAG013, TAG014, TAG015, TAG016, TAG017, TAG018, TAG019, TAG020, TAG0121, TAG022, TAG023, TAG024, TAG025, TAG026, TIE001, TIE002, TAG55WH, TAG-CONTAIN2
<b>Intended Use</b>	Tray Tags are a range of plastic labelling tags which are intended to be used for tracking and tracing purposes.
<b>Range of Applications for Product</b>	For use with trays, baskets and other items for identification purposes.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Resin bonded</li> <li>• Various colors</li> <li>• X-ray detectable</li> <li>• Autoclavable</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use (s)</b>	For use with trays, baskets and other items for identification purposes.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures):</b>	
<b>Steps for Use of Product</b>	<p><b>HOW TO USE TRAY TAGS</b></p> <p><b>Writing directly onto the Tray Tag:</b></p> <ol style="list-style-type: none"> <li>1. Information may be placed onto a Tray Tag by writing directly onto it using a suitable marking PEN. Care should be taken to ensure that the writing will not be removed during reprocessing. It is recommended to test this first, prior to placing in use.</li> <li>2. Either or both sides of the Tag may be written on:</li> </ol> <div style="text-align: center; border: 1px solid black; padding: 10px; margin: 10px 0;">  </div> <p><b>Using a pre-printed self-adhesive label:</b></p> <ol style="list-style-type: none"> <li>1. Pre-printed self-adhesive labels may be stuck onto either side of a Tag. Care should be taken to ensure that the label and its information are compatible reprocessing. It is recommended to test this first, prior to placing in use.</li> </ol>

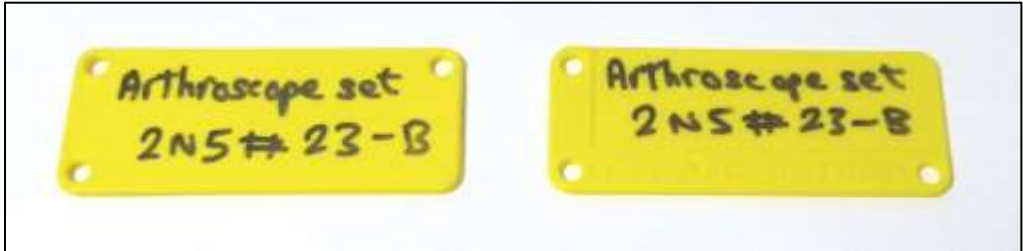

**When Attaching the Tray Tag to Tray or Basket or other items:**

1. Use the holes in the Tray Tag to attach it to the Tray or Basket.
2. Use a split ring (RIN001) or a Security Tie (TIE001 or TIE002) for this purpose.
3. If X-Ray detectability is required, use a split ring, RIN001, to attach the Tray Tag to the tray since the split ring is X-Ray detectable.



<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>● The facility should develop a policy for the placement of the identification tag. This should include a risk assessment to avoid potential interference with effective processing.</li> <li>● Tray Tags should be stored away from any sources of ignition.</li> </ul>
<b>Disposal</b>	<ul style="list-style-type: none"> <li>● Unused Tray Tags may be disposed of in the plastic waste stream.</li> <li>● Used Tray Tags should be disposed of by taking into account possible microbial contamination.</li> <li>● Users should reference facility procedures in this matter.</li> </ul>

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	Tray Tags are compatible with washer-disinfectors conforming to ISO 15883 series of standards.
<b>Disinfection</b>	
<b>Drying</b>	

<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>● Materially compatible with Steam, EO and VHP Sterilization</li> <li>● Recommended cycle for steam sterilization is 270°F for 4 minutes, or 275°F for 3 minutes.</li> </ul>
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	Labeling Brochure, Labeling Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2019-07-16 msmith

Rev. B