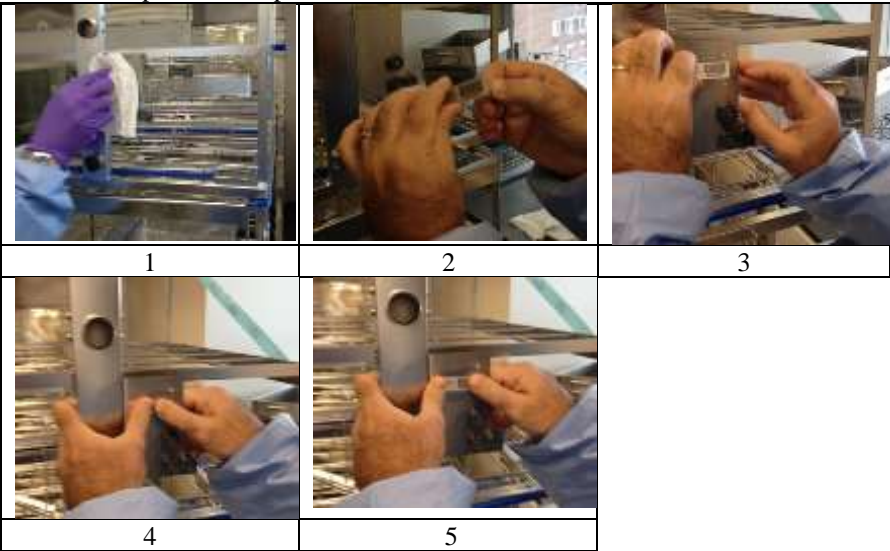


Instructions for Use: Washer Rack Label

Brand Name of Product	Washer Rack Labels
Generic Name of Product	Washer Rack Labels
Product Code Number(s)	WRL-001
Purpose of Product	The Washer Rack Labels are used to identify and track washer racks through the washing cycles when being tested with the TOSI® and other Proformance™ cleaning verification products.
Range of Applications for Product	Washer Racks
Key Specifications of Product	<ul style="list-style-type: none"> • Anodized aluminum • Adhesive backing • Designed to withstand repeated washing cycles

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Like a license plate for your washer rack, each label has a unique human and machine readable alphanumeric number. The Washer Rack Label will help determine washer racks that are repeatedly not performing to cleaning standards so the issue can be properly identified. Manufactured with anodized aluminum and robust adhesive backing, they are designed to withstand repeated washing cycles.
Preparation for Use	<ol style="list-style-type: none"> 1. Wipe surface of the rack clean where you intend on placing the label. 2. Be sure surface is dry prior to adhering label. 3. Be sure your fingers do not come in contact with the adhesive as this will compromise the performance of the adhesive over time.
Diagrams (drawings, pictures)	 <div style="display: flex; justify-content: space-around; margin-top: 5px;"> 1 2 3 </div> <div style="display: flex; justify-content: space-around; margin-top: 5px;"> 4 5 </div>
Steps for Use of Product	<ol style="list-style-type: none"> 1. Wipe washer rack surface clean where the label will be placed. Be sure surface is dry. 2. To remove the label from the wax paper backing, grab hold of the tab area (small area beyond the score)/ 3. Bend the tab area back and forth at the score to loosen the tab from the label. 4. Peel the tab and wax backing from the label. Be sure not to touch the adhesive when doing so. 5. Place the label on the washer rack even as possible without your fingers coming in contact with the adhesive on the back. 6. Press firmly with your thumbs from the middle outward.

	7. Make sure the label is firmly secure to the washer rack and that there is no air between the rack and the label.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	<ul style="list-style-type: none"> • Be sure fingers do not come in contact with the adhesive as this will compromise the adhesive and shorten the useful life of the label. • Do not pull-test the label off for at least three days. It can be used, but it needs three days to properly set.
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	Label Brochure, Label Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com