

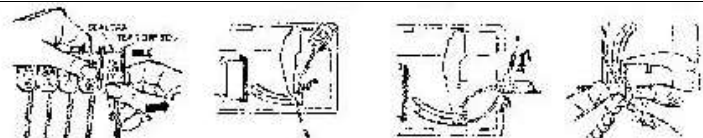
Instructions for Use: Easy Open Pull-Tight Loks

Brand Name of Product	Easy Open Pull-Tight Loks
Generic Name of Product	Tamper Evident Seals
Product Code Number(s)	0345 RD, 0345 WT
Purpose of Product	To show tampering and/or access to critical equipment and supplies.
Range of Applications for Product	Crash carts, emergency boxes, cabinets, and other areas where demonstrating tampering is critical.
Key Specifications of Product	<ul style="list-style-type: none"> • Numbered. • Seal and broken by hand. • Cannot be resealed once broken. • Red or white • Cynch-style allows for adjustment to application lengths

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	100 per box
Shelf Life	

Instructions for Using Product

Description of Use (s)	Seal crash carts, emergency boxes and other areas where evidence of tampering is critical
Preparation for Use	
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Hold seal strip as shown. On seal being detached, place thumb on seal-tab, not on tear off rod. Bend seal-tab and pull apart from strip. 2. Insert bail through locking mechanism to be sealed and into locking barrel of seal. 3. Pull bail tight. 4. Remove by pulling tear-off rod away from seal-tab and out of locking mechanism.
Interpretation of Results	
Contraindications of Test Results	
Documentation	Follow the policy of your facility for recording date, staff and other information critical to management of critical inventory.
Special Warnings and Cautions	If a seal is broken, assume items have been tampered with or removed. Follow your facility instructions for inspection and replacement of critical inventory.
Disposal	

Reprocessing Instructions

Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	Compatible with steam sterilization. Standard cycle: 270°F/4 minutes
Storage	
Additional Information	

Related Healthmark Products	
------------------------------------	--

Other Product Support Documents	Security Products Brochure, Security Products Price List
Reference Documents	Made in USA
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Bvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-02-10 msmith