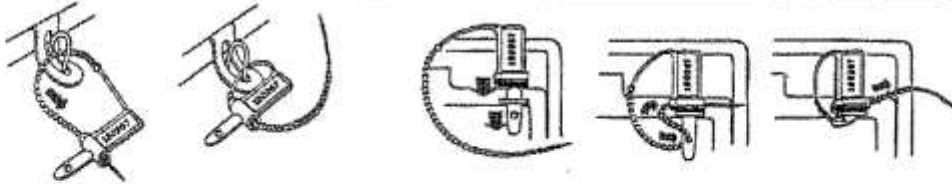


**Instructions for Use: Cynch-Loks**

<b>Brand Name of Product</b>	Cynch-Loks
<b>Generic Name of Product</b>	Tamper Evident Seals
<b>Product Code Number(s)</b>	6020, 6323, 2220 LOK
<b>Purpose of Product</b>	To show tampering and/or access to critical equipment and supplies.
<b>Range of Applications for Product</b>	Crash carts, emergency boxes, cabinets, and other areas where demonstrating tampering is critical.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Numbered or unnumbered</li> <li>• Cynch-style allows for adjustment to application lengths</li> <li>• Seal can be broken by hand</li> <li>• Cannot be resealed once broken</li> <li>• Variety of colors available</li> <li>• 2220 LOK communicates “Clean” and “Biohazard” during transportation</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	100 per box
<b>Shelf Life</b>	N/A

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Seal crash carts, emergency boxes and other areas where evidence of tampering is critical
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<p><b>For Normal Apertures:</b></p> <ol style="list-style-type: none"> <li>1. Detach bail tip from locking barrel. Pass bail through opening of item to be sealed and through tapered end of locking barrel.</li> <li>2. Pull bail tightly all the way through. Tug on seal to ensure it is securely locked.</li> </ol> <p><b>For Narrow Slot Apertures:</b></p> <ol style="list-style-type: none"> <li>1. Detach bail tip from locking barrel. Insert tab into narrow slot.</li> <li>2. Bend tab upwards &amp; pass bail through hole of tab. Twist bail around &amp; insert into tapered end of locking barrel.</li> <li>3. Pull bail through barrel as far as it will go.</li> </ol>
<b>Interpretation of Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	Follow the policy of your facility for recording date, staff and other information critical to management of critical inventory.
<b>Special Warnings and Cautions</b>	If a seal is broken, assume items have been tampered with or removed. Follow your facility instructions for inspection and replacement of critical inventory.
<b>Disposal</b>	N/A

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A

<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Compatible with steam sterilization. Standard cycle: 132°C (270°F) for 4 minutes.</li> <li>• Maximum steam sterilization: 135°C for 30 minutes.</li> </ul>
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	Security Products Brochure, Security Products Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2021-05-11 msmith