




healthmark
INDUSTRIES CO.
health care products

33671 Doreka
Fraser, Michigan 48026
Telephone: (586) 774-7600
Fax: (586) 774-6473
www.HMARK.com


LOOP LOCKS PRODUCT# 9075

INSTRUCTIONS FOR USE

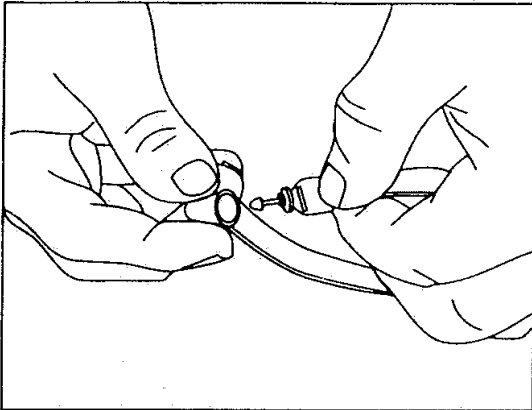


1. Holding dome in one hand, place seal through latch. Grab the free end just below the spear point.

Sujetar el boton con una mano, pasar el sello a través del pestillo. Sujetar el extremo suelto justo debajo de la punta de lanza.

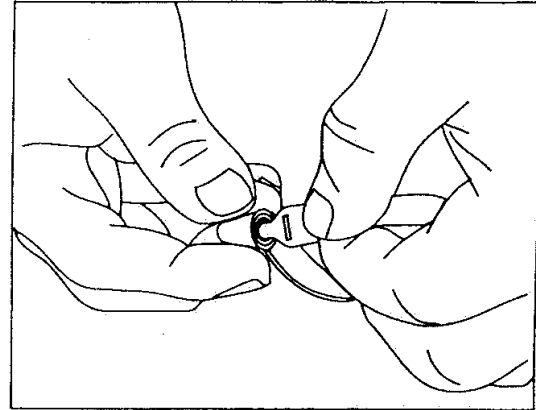

2. Insert the spear point into the dome. Listen and feel for closing snap.

Introducir la punta de lanza en el boton. Escuchar y sentir el chasquido de cierre.

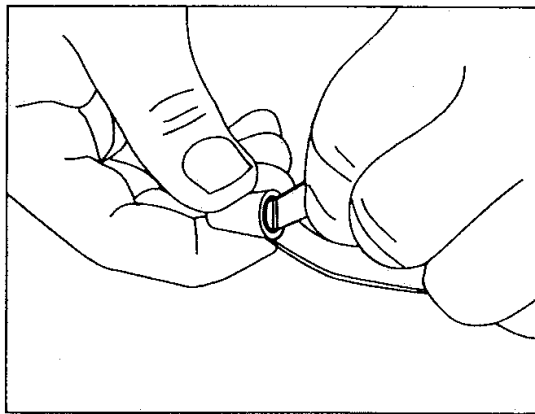

3. Twist and tug on seal to determine secure closure.

Girar y tirar del sello para comprobar el cierre seguro.