



**Instructions for Use: Long Cynch-Loks**

<b>Brand Name of Product</b>	Long Cynch-Loks
<b>Generic Name of Product</b>	Tamper Evident Seals
<b>Product Code Number(s)</b>	927-15, 927-07, 927-11, 1515 GN HLD
<b>Intended Use</b>	To show tampering and/or access to critical equipment and supplies.
<b>Range of Applications for Product</b>	Crash carts, emergency boxes, cabinets, and other areas where demonstrating tampering is critical.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Numbered</li> <li>• Seal and broken by hand</li> <li>• Cannot be resealed once broken</li> <li>• Variety of colors available</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Seal crash carts, emergency boxes, and other areas where evidence of tampering is critical.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Place strap through locking chamber with teeth towards neck of bag.</li> <li>2. Pull strap manually or place hook of puller tool in slotted strap-end. Pull tightly.</li> <li>3. For tight grip, place jaws of the tool over teeth and pull. The bag is now securely sealed.</li> <li>4. To remove surplus strap, place jaws of tool over teeth at locking chamber and twist strap.</li> </ol>
<b>Interpretation of Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	Follow the policy of your facility for recording the date, staff, and other information pertinent to management of critical inventory.
<b>Special Warnings and Cautions</b>	If a seal is broken, assume items have been tampered with or removed. Follow your facility's instructions for inspection and replacement of critical inventory.
<b>Disposal</b>	N/A

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A

<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	Compatible with steam sterilization. Standard cycle: 132 °C (270 °F) for 4 minutes
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	Security Products Brochure, Security Products Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com