


Instructions for Use: Plastic & Wire Loks

Brand Name of Product	Plastic & Wire Loks
Generic Name of Product	Tamper Evident Seals
Product Code Number(s)	PWR-6
Purpose of Product	To show tampering and/or access to critical equipment and supplies.
Range of Applications for Product	Crash carts, emergency boxes, cabinets, and other areas where demonstrating tampering is critical.
Key Specifications of Product	<ul style="list-style-type: none"> • Numbered • Seal and broken by hand • Cannot be resealed once broken • Variety of colors • Manufactured of tough plastic body and metal wire hasp • Write-on panel on one or two sides

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	100 per box
Shelf Life	

Instructions for Using Product

Description of Use (s)	Seal crash carts, emergency boxes and other areas where evidence of tampering is critical
Preparation for Use	
Diagrams (drawings, pictures):	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Pull on wire to remove loose hasp end from seal body 2. Pass open end through opening of object to be sealed 3. Re-insert hasp end into slot seal body. Firmly push down hasp until it is securely sealed. Pull on seal to ensure locking mechanism is engaged. 4. Twist seal body to break and remove.
Interpretation of Results	
Contraindications of Test Results	
Documentation	Follow the policy of your facility for recording date, staff and other information critical to management of critical inventory.
Special Warnings and Cautions	If a seal is broken, assume items have been tampered with or removed. Follow your facility instructions for inspection and replacement of critical inventory.
Disposal	

Reprocessing Instructions

Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	

Storage	
Additional Information	
Related Healthmark Products	
Other Product Support Documents	Security Products Brochure, Security Products Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-02-10 msmith