



Instructions for Use: Pull Tight-Loks

<b>Brand Name of Product</b>	Pull Tight-Loks
<b>Generic Name of Product</b>	Tamper Evident Seals
<b>Product Code Number(s)</b>	9393, 5030, 8323
<b>Purpose of Product</b>	To show tampering and/or access to critical equipment and supplies.
<b>Range of Applications for Product</b>	Crash carts, emergency boxes, cabinets, and other areas where demonstrating tampering is critical.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Numbered or unnumbered</li> <li>• Seal and broken by hand</li> <li>• Design enables sealing items of various dimensions by pulling Lok to the optimal seal length.</li> <li>• Cannot be resealed once broken</li> <li>• Variety of colors available</li> </ul>

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	100 per box
<b>Shelf Life</b>	N/A

Instructions for Using Product	
<b>Description of Use(s)</b>	Seal crash carts, emergency boxes, and other areas where evidence of tampering is critical.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. To separate seals, hold the tab between thumb and forefinger and pull down. <b>(Fig. 1).</b></li> <li>2. Insert the thin end of the Pull Tight seal stem through the locking dome on the side marked “Enter” and pull the stem through. <b>(Fig. 2).</b> <ol style="list-style-type: none"> <li>a. Pull it as tight as possible around item being sealed.</li> <li>b. This will ensure the seal cannot be broken and reinserted into the locking mechanism. <b>(Fig. 3).</b></li> </ol> </li> <li>3. Utilize the “tug” test by grasping the tab of the seal and gently tug on it to determine closure. <b>(Fig. 4).</b></li> <li>4. If you can loosen the seal, <b>IT’S NOT FULLY LOCKED.</b> Remove the seal and replace it with a new one.</li> </ol>
<b>Interpretation of Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	Follow the policy of your facility for recording date, staff, and other information critical to management of critical inventory.
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• If a seal is broken, assume items have been tampered with or removed. Follow your facility’s instructions for inspection and replacement of critical inventory.</li> <li>• Inserting the Pull Tight seal stem into the wrong side of the tab may cause damage to the locking dome’s teeth and the seal will not work properly. If inserted wrong, remove the seal and begin with a new seal.</li> <li>• <i>Do not reuse a broken seal.</i></li> </ul>
<b>Disposal</b>	N/A

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, inspection, and testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	Compatible with steam sterilization. Standard cycle: 132 °C (270 °F) for 4 minutes
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	Security Products Brochure, Security Products Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>