



Brand Name of Product	STERIKING® Sterilization Tapes																				
Generic Name of Product	Sterilization Tapes																				
Product Code Number(s)	ITE19, ITS12G, ITS19G, ITS25G, Tape Dispenser, Tape Dispenser-2, Tape Dispenser-3																				
Purpose of Product	STERIKING® Steam autoclave tapes are manufactured for closing packages of items requiring steam sterilization in hospitals and other similar uses.																				
Range of Applications for Product	Instrument wraps and other items that need to be taped closed during sterilization.																				
Key specifications of product	<p>STERIKING® Steam autoclave tapes with steam indicator are produced by using strong impregnated crepe paper, coated with an adhesive. This is principally based on natural dry rubber and tackifying synthetic resins.</p> <table border="1"> <thead> <tr> <th>Property</th> <th>Units</th> <th>Typical Value</th> </tr> </thead> <tbody> <tr> <td>Thickness:</td> <td>μ</td> <td>140-170</td> </tr> <tr> <td>Adhesion to steel:</td> <td>N/25 mm</td> <td>9.0 – 10.0</td> </tr> <tr> <td>Tensile strength:</td> <td>N/25 mm</td> <td>70 - 90</td> </tr> <tr> <td>Elongation:</td> <td>%</td> <td>7</td> </tr> <tr> <td>Color:</td> <td></td> <td>Natural, indicator tape printed with white stripes</td> </tr> </tbody> </table>			Property	Units	Typical Value	Thickness:	μ	140-170	Adhesion to steel:	N/25 mm	9.0 – 10.0	Tensile strength:	N/25 mm	70 - 90	Elongation:	%	7	Color:		Natural, indicator tape printed with white stripes
Property	Units	Typical Value																			
Thickness:	μ	140-170																			
Adhesion to steel:	N/25 mm	9.0 – 10.0																			
Tensile strength:	N/25 mm	70 - 90																			
Elongation:	%	7																			
Color:		Natural, indicator tape printed with white stripes																			

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	It is recommended that the STERIKING® products are kept in the original, closed transport cartons and stored in dry and clean conditions protected from direct sunlight and excessive moisture and heat.
Packaging Conditions	<p>STERIKING® steam autoclave tapes are packed in recyclable, brown hard carton cases. As an inner cover against moist and dust the steam indicator tapes are wrapped in a recyclable HD polyethylene sheet.</p> <p>Labelling: Each case bears a label with the necessary information/instructions of the content in accordance with ISO 11140-1:2005.</p>
Shelf Life	If stored according to instructions the shelf life of STERIKING® Steam autoclave tapes are max. 4 years from the date of manufacture.


Instructions for Using Product

Description of Use (s)	
Preparation for Use	Place new roll of tape in the dispenser.
Diagrams (drawings, pictures)	

Starting color of unexposed indicator of ITSG-tape:



End colors of correctly exposed indicator of ITSG-tape:

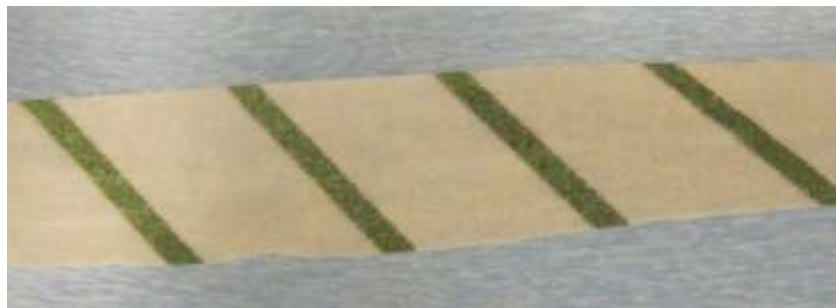

Please note:

The Striking ITSG indicator tapes' color change may vary due to differences in processing parameters. The indicators should not be exposed to direct light prior to use.

Steps for Use of Product

- Use tape to close packages and items for sterilization.
- Indicator (Class 1) tapes with STEAM indicator changing color from white to brown / black.
- Indicator tapes with EtO gas indicator changing color from browns to green.

Interpretation of Results

Contraindications of Test Results

Documentation

Special Warnings and Cautions

Disposal

Reprocessing Instructions

Point of Use

Preparation for Decontamination

Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	<p><u>Steam</u> The steam indicator ink will react positively when subjected to heat and moisture in the steam autoclave. After sterilization strong dark lines appear on the backing. The usual time/temperature combinations are 3 minutes at 273°C or 15 minutes at 249°C.</p> <p><u>EO</u> The ethylene oxide indicator ink will react positively when subjected to ethylene oxide autoclave process.</p>
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	Sterilization Price List, Sterilization Brochure
Reference Documents	STERIKING® Steam indicator tapes conform to the international standard: ISO 11140-1 2005 (Class 1, process indicators).
Customer Service Contact	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-11-05 Ralph J Basile