


**Instructions for Use: Slotted Tray label IFU**

<b>Brand Name of Product</b>	Slotted Tray Label
<b>Generic Name of Product</b>	Slotted tray label
<b>Product Code Number(s)</b>	403225 SLOTTED, 403225 SLOTTED PINK
<b>Purpose of Product</b>	For placement of instrument count sheets on the outside of a closed sterilization container in compliance with AORN Recommended Practice.
<b>Range of Applications for Product</b>	Instrument Sterilization Containers
<b>Key specifications of product</b>	<ul style="list-style-type: none"> <li>• Self-looping style for mounting on the handle of any closed container</li> <li>• Compatible with steam, EO and gas-plasma sterilization</li> <li>• Slotted to allow for convenient and robust placement of instrument count sheets</li> <li>• Available in 2 colors</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Conditions</b>	Packaged 500 per box
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use (s)</b>	The slotted tray label allows you to attach your surgical instrument count sheet on the outside of a closed sterilization container. It eliminates the concern of ink transfer from the count sheet to any instrument.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures):</b>	 <p style="text-align: center;"> <span style="margin-right: 20px;"><i>Figure 1</i></span> <span style="margin-right: 20px;"><i>Figure 2</i></span> <span style="margin-right: 20px;"><i>Figure 3</i></span> <span><i>Figure 4</i></span> </p>
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. The slotted tray label has two slots to hold the rolled up count sheet (figure 1).</li> <li>2. After you have secured (closed, locked) the container; you will loop the slotted tray tag through the handle of the container (figure 2).</li> <li>3. Take the surgical count sheet and roll it (when possible with the ink on the inside of the roll) and then place it between the two slots (figure 3).</li> <li>4. Push the count sheet through the slotted tray label so it is evenly hanging on the slotted tray label. You can also affix an ID label to the front of the tag (figure 4).</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

<b>Reprocessing Instructions</b>	
<b>Point of use:</b>	
<b>Preparation for decontamination:</b>	
<b>Disassembly Instructions:</b>	
<b>Cleaning – Manual:</b>	
<b>Cleaning – Automated:</b>	
<b>Disinfection:</b>	
<b>Drying:</b>	
<b>Maintenance, inspection, and testing:</b>	
<b>Reassembly Instructions:</b>	
<b>Packaging:</b>	
<b>Sterilization:</b>	Compatible with steam, EO and gas-plasma/VHP sterilization

<b>Storage:</b>	
<b>Additional Information:</b>	
<b>Related Healthmark Products</b>	PBR-46 Medical Grade Paper, PBR 76 Count Sheet Holder with Poly from and medical grade back
<b>Other Product Support Documents</b>	Steriking Price List, Sterilization Brochure, Labels Brochure, Labels Price List
<b>Reference Documents</b>	AORN 2011 Recommended Practice for Packaging Systems: Recommendation IV: P3
<b>Customer Service contact:</b>	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2016-08-01 msmith