



Instructions for Use: Slotted Tray label IFU

Brand Name of Product	Slotted Tray Label
Generic Name of Product	Slotted tray label
Product Code Number(s)	403225 SLOTTED, 403225 SLOTTED PINK
Purpose of Product	For placement of instrument count sheets on the outside of a closed sterilization container in compliance with AORN Recommended Practice.
Range of Applications for Product	Instrument Sterilization Containers
Key Specifications of Product	<ul style="list-style-type: none"> • Self-looping style for mounting on the handle of any closed container • Slotted to allow for convenient and robust placement of instrument count sheets • Available in 2 colors

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	Packaged 500 per box
Shelf Life	

Instructions for Using Product	
Description of Use (s)	The slotted tray label allows you to attach your surgical instrument count sheet on the outside of a closed sterilization container. It eliminates the concern of ink transfer from the count sheet to any instrument.
Preparation for Use	
Diagrams (drawings, pictures)	<div style="display: flex; justify-content: space-around; margin-top: 5px;"> Figure 1 Figure 2 Figure 3 Figure 4 </div>
Steps for Use of Product	<ol style="list-style-type: none"> 1. The slotted tray label has two slots to hold the rolled-up count sheet (Figure 1). 2. After you have secured (closed, locked) the container; you will loop the slotted tray tag through the handle of the container (Figure 2). 3. Take the surgical count sheet and roll it (when possible with the ink on the inside of the roll) and then place it between the two slots (Figure 3). 4. Push the count sheet through the slotted tray label so it is evenly hanging on the slotted tray label. You can also affix an ID label to the front of the tag (Figure 4).
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	Compatible with steam, VH2O2/O3/EO sterilization
Storage	

Additional Information	
Related Healthmark Products	PBR-46 Medical Grade Paper, PBR 76 Count Sheet Holder with Poly front and medical grade back
Other Product Support Documents	Steriling Price List, Sterilization Brochure, Labels Brochure, Labels Price List
Reference Documents	AORN 2011 Recommended Practice for Packaging Systems: Recommendation IV: P3
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com