



Instructions for Use: Dual UnderGuard™ Dry Mats

Brand Name of Product	Dual UnderGuard™ Dry Mats
Generic Name of Product	Dry Mats
Product Code Number(s)	DRYMAT001, DRYMAT001-GRN, DRYMAT002, DRYMAT002-GRN, DRYMAT003, DRYMAT003-GRN, DRYMAT004, DRYMAT011, DRYMAT012, DRYMAT018-GRN, DRYMAT018, DM-2030PP, DM-2030GRN, DM-1520PP, DM-1520GRN, DM-1224PP, DM-1224GRN, DM-1020PP, DM-1220PP, DM-0820PP, DM-121GN, DM-1212PP.
Purpose of Product	To be used under trays to protect against handling while simultaneously providing an absorbent inner layer for the dispersion of condensate during the autoclaving process.
Range of Applications for Product	Provides protection useful for heavy instrument trays and baskets.
Key Specifications of Product	Dry mats are made from polypropylene and viscose.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	+15°C-30°C and 30-70% RH
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Strong spun polypropylene sheet with an absorbent layer for use as a strong mat under trays-to protect against sharp-edged trays and provide rapid moisture dispersion during steam sterilization.
Preparation for Use	
Diagrams (drawings, pictures)	
Steps for Use of Product	<ul style="list-style-type: none"> • Place liner under tray to protect from tears caused by handling. • Place liner inside tray to provide absorbent inner layer for the dispersion of condensate during autoclaving process. <ol style="list-style-type: none"> 1. Place items in tray 2. Use proper sterility assurance indicators and follow hospital protocol 3. Wrap close package 4. Place in steam sterilizer
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	Dry mats are not suitable for use in Plasma sterilizing systems.
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	<ul style="list-style-type: none"> • Steam sterilization at 270°F (132°C) for 4 minutes or 275°F (135°C) for 3 minutes. • Gravity steam sterilization at 250°F(121°C) for 30 minutes • Maximum extended time for 30 minutes at 275°F (135°C)

	<ul style="list-style-type: none"> • Compatible with EO sterilization.
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-07-05 msmith