





Instructions for Use: UnderGuard™ Tray Liners

Brand Name of Product	UnderGuard™ Tray Liners
Generic Name of Product	Paper Tray Liners
Product Code Number(s)	TL-0812, TL-1010, TL-1020, TL-1216, TL-1822, TL-1212, TL-1218, TL-1224, TL-1226, TL-1818, TL-1824, TL-0610, TL-0624, TL-0806, TL-1120, TL-1117, TL-0911, TL-0711, TL-0509, TL-1422, BL-1414, BL-1419, BL-1423.
Intended for Use	To line the bottom of sterilization trays and containers during steam sterilization to absorb and rapidly disperse condensate to facilitate complete evaporation. The liners can be safely used inside trays and placed next to delicate instruments for steam sterilization or can be used underneath trays. In both cases, condensate dispersion facilitates the production of dry packs. The tray liners may also be used as a packaging accessory on the inside of wrapped sterile packages or sterilization containers to protect or separate items and provide wicking.
Range of Applications for Product	For use in sterilization trays, including wrapped and closed containers.
Key Specifications of Product	<ul style="list-style-type: none"> • Single-use • Material made from pure cellulose EFG wood pulp, manufactured with low chloride and sulfate content and free optical dyes • Non-Linting • Use facilitates the production of dry packs • Tray liners provide protection for delicate and sharp instruments • Form fitting tray liners also prevent instruments from protruding through openings on the bottom and sides of trays and baskets

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	+15°C-30°C and 30-70% RH
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use(s)	To line the bottom of sterilization trays and containers during steam sterilization to absorb and rapidly disperse condensate to facilitate complete evaporation.
Preparation for Use	<ol style="list-style-type: none"> 1. Select a liner which will best fit the target basket. 2. If using with wrapped tray, liner must be inside the wrap and may be used inside or under the basket. 3. If using with a closed container, liner must be used inside the container, either inside or below the basket.
Diagrams (drawings, pictures)	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>Inside Tray</p> </div> <div style="text-align: center;">  <p>Beneath Tray</p> </div> </div>
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place liner either inside or beneath basket. In the event you are using a container, the liner should be placed in case first. 2. Place items in tray. 3. Use proper sterility assurance indicators and follow hospital protocol. 4. Wrap or close package 5. Place in steam or Ethylene Oxide sterilizer
Interpretation of Results	

Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	Single-use. Dispose of in compliance with facility policy.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	<ul style="list-style-type: none"> ● Steam sterilization at 132°C for 4 minutes or 135°C for 3 minutes ● Gravity steam sterilization at 121°C for 30 minutes ● Maximum extended time for 30 minutes at 135°C ● Compatible with EO sterilization
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2020-08-04 msmith