




Instructions for Use: UnderGuard™ for Autoclave Shelves

Brand Name of Product	Underguard™ for Autoclave Shelves
Generic Name of Product	Tray Liners
Product Code Number(s)	SOK008SS, SOK009SS, SOK010SS, SK-2248, SK-2260, SK-2036.
Purpose of Product	For lining the shelves of steam sterilizer carts to protect against the tearing of wrap and to reduce the incidence of wet pack.
Range of Applications for Product	Liners fit both floor loaded and shelf loading autoclaves.
Key Specifications of Product	<ul style="list-style-type: none"> • Material made from pure cellulose paper. • These sheets serve to absorb and rapidly disperse condensate during sterilization. • Helps to prevent dripping of condensate between shelves. • Protect wrapped trays from any sharp edges that might tear wrap when trays are loaded or unloaded.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	+15°C-30°C and 30-70% RH
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	For lining the shelves of steam sterilizer carts to protect against the tearing of wrap and to reduce the incidence of wet pack.
Preparation for Use	
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Extend tray liner to both ends of the shelves on the autoclave loading cart. 2. May be used for multiple cycles. 3. Change liners as needed based on wear, tear and discoloration 4. Between cycles, check each liner for tears and discolorations. If either is observed, dispose of and replace with a new sheet.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	<ol style="list-style-type: none"> 1. Change liners as needed based on wear, tear and discoloration 2. Between cycles, inspect liner for tears and discolorations. If observed, dispose of and replace with a new liner.
Reassembly Instructions	
Packaging	

Sterilization	<ul style="list-style-type: none"> ● Steam sterilization at 270°F (132°C) for 4 minutes or 275°F (135°C) for 3 minutes. ● Gravity steam sterilization at 250°F (121°C) for 30 minutes. ● Maximum extended time for 30 minutes at 275°F (135°C). ● Compatible with EO sterilization.
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-07-05 msmith