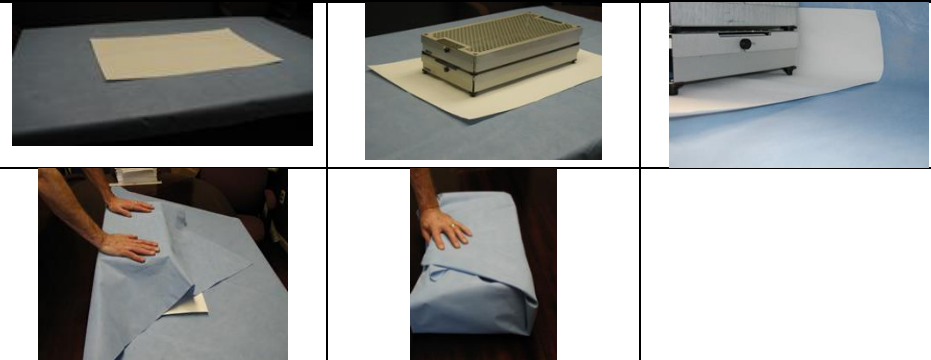


Brand Name of Product	OrthoShield™ Soaker Sheets
Generic Name of Product	Paper Soaker Sheets
Product Code Number(s)	SOK001SS, SOK002SS, SOK003SS, SOK006SS, SOK014SS, SK-2030, SK-1520, SK-1224, SK-1010, SK-1626.
Purpose of Product	Specifically developed for heavy duty use, most commonly under trays to protect wraps from sharp corners and uneven surfaces of trays and baskets while also providing rapid absorption of condensate during sterilization.
Range of Applications for Product	For heavy orthopedic sets, which often present the greatest challenge to the integrity of packaging and are most prone to the occurrence of wet pack.
Key Specifications of Product	<ul style="list-style-type: none"> • Single-use • Material made from pure cellulose EFG wood pulp, manufactured with low chloride and sulfate content and free optical dyes. • Use greatly facilitates the production of dry packs.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	+15°C-30°C and 30-70% RH
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Use appropriate size wrapper for tray selected.
Preparation for Use	
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place soaker sheet on top of wrap. 2. Center where tray will be placed. 3. Place tray on top of soaker sheet (centered). 4. Wrap tray following standard technique. 5. Follow steam sterilization instructions provided by the device manufacturer.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	Single-use. Dispose of in compliance with facility policy for disposing of all sterilization packaging.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	

Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	<ul style="list-style-type: none"> ● Steam sterilization at 270°F (132°C) for 4 minutes or 275°F (135°C) for 3 minutes. ● Gravity steam sterilization at 250°F (121°C) for 30 minutes. ● Maximum extended time for 30 minutes at 275°F (135°C). ● Compatible with EO sterilization.
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 33671 Doreka Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-07-05 msmith