



## Product Technical Data Sheet

Form No.: FTP-004 Rev.: G

**Product Name:** SteriTec Cross-Checks Indicator

**Product Type:** Chemical Indicator

**Reorder Number:** CI 107

**Description:** The SteriTec Cross-Checks is multi-variable indicator. It gives visual confirmation of the attainment of sterilization conditions in sterilizers operating at 132° – 134°C for 3 minutes or longer at the location of the indicator strip within the sterilizer chamber.

### Performance Characteristics:

When all steam sterilization conditions are met, the check marks change color.

**Color Change:** White to Black

**Critical Parameters:** *(in a standard hospital steam sterilizer)*

3 minutes +/- .5 min at 132° - 134°C

**Stated Values:** *(as demonstrated in a steam resistometer)*

3 minutes (+0/-1.25 min) at 132°C (+0/-2°C)

2.5 minutes (+0/1.125 min) at 134°C (+0/-2°C)

**Indicator Stability:** Before and After Sterilization = Excellent

### Physical Characteristics:

**Dimensions:** 0.5625" W x 7.66" L

**Substrate Material:** 80 pound Cover

**Quantity / Bag:** 250

**Bags / Case:** 16

**Selling Unit:** Case

### Environmental Limitations:

**Storage:** 50° - 100°F (10° – 38°C) at 10 - 90% humidity

**Shelf Life:** The expiry date is printed on the product packaging.

**FDA 510(k) Release to Market Date:** August 19, 1996, K951113

### Standards Classifications:

ANSI/AAMI ST79, Class 4

ISO 11140-1, Class 4

DISTRIBUTED BY:  
HEALTHMARK INDUSTRIES, CO.  
33671 DOREKA DRIVE  
FRASER, MI 48026  
1-800-521-6224 / WWW.HMARK.COM