



Container Card

REF CI 138

Directions For Use

INDICATIONS FOR USE

SteriTec Container Card chemical indicators are designed for use in steam sterilizers operating at 121°C to 134°C (250° F to 273° F). When used as directed, the SteriTec Container Card indicator gives visible confirmation that the container was exposed to steam.

COLOR CHANGE

Following sterilization, the indicator turns from white to black.

INSTRUCTIONS FOR USE

1. Complete appropriate information on Container Card.
2. Slide a SteriTec Container Card in each container card holder for sterilization. Adjust the card size by folding along the perforated top edge.
3. Sterilize container as usual in a steam sterilizer.
4. Following sterilization cycle, verify that indicator bar has turned from white to black.

SAFETY PRECAUTIONS

Caution: Do not use cards when indicator bar is not white in color prior to exposure to steam sterilization processing.

STORAGE

The SteriTec Container Cards should be stored at room temperature 50°-100° F (10°-38° C) and normal humidity (10-70% R.H.).

EXPIRY DATE

The expiry date is printed on the product packaging.

DISTRIBUTED BY:
HEALTHMARK INDUSTRIES, CO.
33671 DOREKA DRIVE
FRASER, MI 48026
1-800-521-6224 / WWW.HMARK.COM