

US Steri-Pak LF Bowie-Dick Test Pack BD 111

The Steri-Pak LF Bowie-Dick Test Pack is designed for daily monitoring of pre-vacuum steam sterilizers operating at 270° F (132°C) for 3.5 minutes to detect the presence of residual air.



Key Features

- Complete color change - 3.5 minutes at 132°C
- Blue color remains if an air leak is present
- Lead and toxic heavy metal free

Performance Characteristics

If the sterilizer is functioning properly, the indicator will show a uniform color change (black) throughout the entire pattern. When a significant amount of residual air is present, the indicator shows a blue area in the center.

Physical Characteristics

- Dimensions Card: L 125mm x W 95mm
- Dimensions Pack: L 128mm x W 98mm x D 17mm

Storage

The storage conditions are on the product packaging. Protect from moisture and excess humidity. Store exposed indicator sheet away from light sources.

Shelf Life

36 months from date of manufacture. The expiry date is printed on the product packaging.

510(K)

K990246

Standards Classification

- FDA Cleared Air Removal Indicator
- ANSI/AAMI/ISO 11140-5

SteriTec Part No.

BD 111, US Steri-Pak LF
Bowie-Dick Test Pack

Packaging

20 test packs

FTP-082 Rev. G