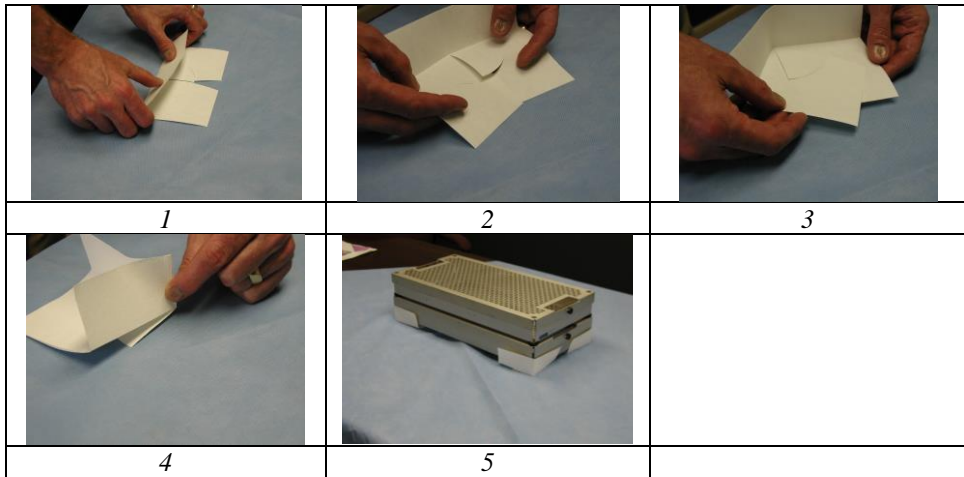


Instructions for Use: UnderGuard™ Tray Corners

Brand Name of Product	UnderGuard™ Tray Corners
Generic Name of Product	Paper Tray Corners
Product Code Number(s)	CC001, CC001 ASM, CC002, CC002 ASM, CC003, TC-001, TC-001ASM, TC-002, TC-002ASM, TC-003.
Purpose of Product	To protect the corner of sterilization, wrap from the sharp edges of sterilization trays.
Range of Applications for Product	Wrapped sterilization trays that are steam sterilized.
Key Specifications of Product	Medical grade cellulose: <ul style="list-style-type: none"> • TC-CC001: Card Stock • TC-CC002: Crepe Stock • TC-CC003: Flat Crepe Stock • TC-ASM (001 & 002) Assembled Tray Corners

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	+15°C-30°C and 30-70% RH
Packaging Conditions	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	To protect the corner of sterilization wrap from the sharp edges of sterilization trays.
Preparation for Use	
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1) Fold 2) Tuck 3) Tuck 4) Pinch 5) Place on corners of the tray.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	Single-use. Dispose of in compliance with facility guidelines for disposal of sterilization packaging material.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	

Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	<ul style="list-style-type: none"> ● Steam sterilization at 270°F (132°C) for 4 minutes or 275°F (135°C) for 3 minutes. ● Gravity steam sterilization at 250°F (121°C) for 30 minutes. ● Maximum extended time for 30 minutes at 275°F (135°C). ● Compatible with EO sterilization.
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	Sterilization Price List, Sterilization Brochure
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 33671 Derek Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2018-07-05 msmith