

INSTRUCTION FOR USE FLAT & FORM FITTING TRAY LINERS

*Produced from pure medical grade cellulose these **single use** liners can be safely used close to delicate instruments during sterilization to absorb moisture and rapidly disperse condensate to facilitate complete evaporation and drying. Tray liners provide protection of delicate and sharp instruments. Form fitting tray liners also prevent instruments from protruding through openings on the bottom of trays and baskets.*

- 1) Once you've chosen the appropriate sterility maintenance material (container, wrap or pouch) place liners either inside or beneath basket. In the event you are using a container the liner should be placed in case 1st, then basket.**



Inside Tray



Beneath Tray

- 2) Place items in tray.**
- 3) Use proper sterility assurance indicators and follow hospital protocol.**
- 4) Wrap or close package**
- 5) Place in Steam Sterilizer**